Rollator Torro

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Stand: 01/2015

Indication / Contraindication

Walking device – for indoor and outdoor use – for compensation and improvement respectively of restricted ability of walking.

The purpose of walking devices is to compensate for or at least improve restricted walking ability in users (expansion of range and/or mobility).

Persons with restricted hand/arm functions require special adjustments/versions of handgrips or hand rests/armrests.

For persons with balance disorders walking frames are not recommended, this is to be checked by an assessment in an individual case. If it is necessary a companion must be present.

Contraindication


Use of the Rollator is not recommended for persons suffering from: perceptual disorders, severe imbalance disorders, an inability to sit

Scope of delivery contents

1. Rollator
2. Bag
3. Backrest
4. Cane holder
5. Operating instructions


Unfold

Position the Rollator on its wheels. Open the red safety bow and lift up. Now press both pipes from the seat downwards until they interlock. (see pictures 1 - 2).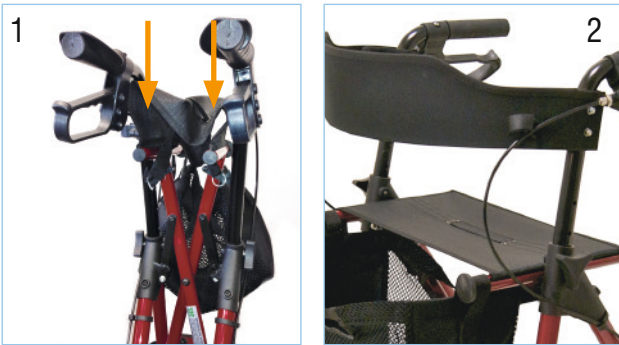

Fitting the bag


Hang the bag with the Velcro loops at the frame tube ends. (see pictures 4 - 6).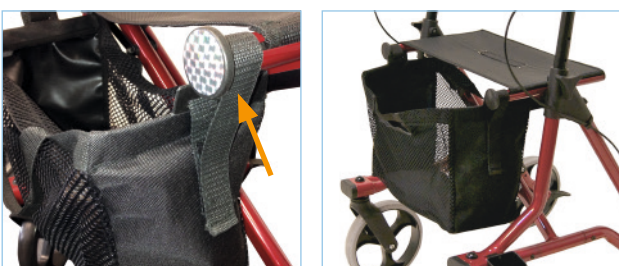

Height adjustment handgrips

Insert the tube of the adjustable handgrip into the frame at least to the "first hole". Insert the locking screw through the pipe from the inside and fix with washer and star clutch screw. (see picture 3).

Turn the star clutch counterclockwise to release the locking screw and change the height setting of the adjustable handgrips. Then follow the above instructions to secure the handgrip at the desired height. (see picture 4).

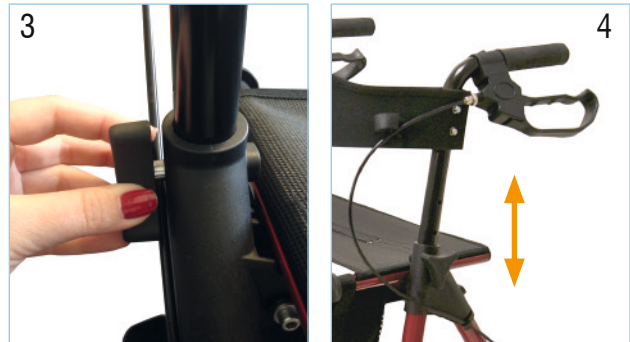Adjustment of brake

For desired braking strength adjust the brake either manually at the screw on the housing of the brake (see picture 5) or with the suitable tool (not included) on the brake wire (see picture 6).

Breaking / Parking

Pull both brake levers upwards to apply the brakes whilst the Rollator is moving (see picture 7). Always brake slowly and steady.

Press both brake levers down to apply the parking brakes whilst the Rollator is stationary (picture 8).

The braking effect of the parking brake acting on the rear wheels can be metered and depends on force applied.

Pull up both brake levers in order to release the parking brakes (the levers are in the middle position).

The effect of the brakes can be affected by abrasion of the wheels. **Therefore regularly test the brakes.**

Sitting

Position the Rollator on an even and solid surface. Then lock the brakes (parking position). The Rollator must stand and must not slip. Sit straight up and do not lean backwards (see picture 9).

Folding / Transport / Storage

To fold, pull seat upwards by the handle on the seat plate. The Rollator can now be folded. (picture 10 -11).

Safety instructions

Use only as a walking aid!

Use only on even and solid surfaces.

The Rollator must not be used for standing on or as a wheelchair! It must not be used for transport of persons or animals.

Maximum user weight: 150 kg (23 stone)

The use on escalator`s is expressly forbidden!

Regularly check the screws and fasteners on the Rollator and retighten if necessary.

Use the bag only to transport objects and do not overload. Maximum load: 5 kg

Do not expose the product to temperatures above 40°C for a long time, as this can cause the Rollator parts to overheat and burns to the user.

Cleaning / Maintenance

Cleaning

Clean regularly by hand with a damp cloth!
Do not use any aggressive detergents!

Maintenance

Ensure that all parts of the Rollator are functioning correctly. Check all screws and ensure that they are all tightened correctly. Check that all of the push-on and quick-release connectors are positioned correctly and locked in place. Check that all buttons on the push-in and quick release connectors release and re-lock easily. Broken, damaged and used parts must be replaced immediately.

Reprocessing

The Rollator is suitable for reuse.

Ensure that the new user is given all of the technical documents required to ensure safe handling.

The Rollator must be cleaned and disinfected prior to reuse and must be inspected for damage and approved by the retailer.

Warranty

We issue a warranty of 24 months from date of purchase. Subject to technical alterations. Errors and omissions excepted.

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