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Umbrella holder

Porte-parapluie

Parapluhouder

CE

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
Preassembly of the holder:

The following are needed:

1 x	mounting clamp A (20 mm)
1 x	mounting bracket K
1 x	star grip J
1 x	screw G
1 x	screw H
1 x	nut I

Mount the clamp onto the mounting bracket using both screws. The size of the clamp can be read on the inside of the clamp.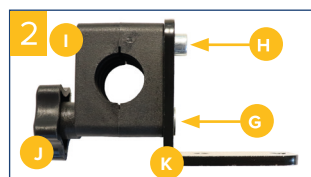


Assembly on the rollator with round tube:

The following are needed:

1 x	mounting clamp A
2 x	screw B

Measure the diameter of the round tube and select the appropriate clamp A. The size of the clamp can be read on the inside of the clamp.

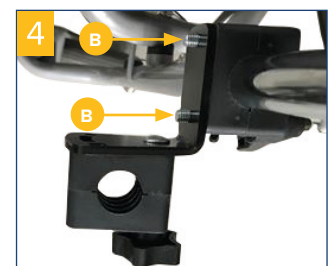Assembly on Nitro rollator (frame assembly):

The following are needed:

1 x	mounting clamp C (Notch in the hole)
2 x	screw E
2 x	protective sleeve F

Assembly on Nitro rollator (push handle assembly):


The following are needed:

1 x	mounting clamp C (Notch in the hole)
2 x	screw D
2 x	protective sleeve F

Insert the screws through the clamp (1) and put the protective sleeve (2) over the screws.

Now secure the pre-assembled holder (see Pre-assembly of the holder) using the clamp to the tube of the rollator.

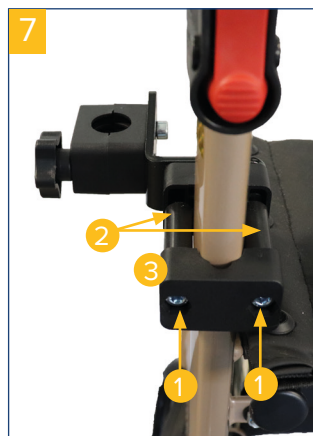Securing the umbrella to the umbrella holder:

Slightly loosen star screw J and insert umbrella holder L into the clamp. Using clip M, you can adjust the desired height by inserting clip M into umbrella holder L (clip M must be positioned above the clamp). Tighten star screw J again.

Insert the umbrella into umbrella holder L and secure it using connecting screw N.

Important: The clearance holes must be aligned.

Angle adjustment

To adjust the angle, press the button and hold it down. Then adjust the desired angle and release the button. The button must audibly and visibly snap into place.

Opening the umbrella

Remove umbrella cover. To open, press the button on the lower end of the rod. The umbrella opens automatically. To close, pull the slider back down. Cover the umbrella cover if necessary (the umbrella must be dry).

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Drive Medical GmbH & Co. KG