# NITRO GLIDE

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## Table of contents

Preface .....	11
Declaration of conformity .....	11
Service life .....	11
Disposal .....	11
Identification plate .....	11
Reuse .....	11
Warranty .....	11
Parts of the product .....	12
Purpose .....	13
Indication .....	13
Contraindication .....	13
Scope of delivery .....	13
Safety instructions .....	13
Assembly .....	14
Height adjustment .....	14
Adjustment of the front axles .....	15
Setting brake .....	15
Braking/ Parking .....	15
Fold/ Transport/ Stowing .....	15
Cleaning   Maintenance .....	16
Technical data .....	16

## Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

## Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co. KG declares compliance according to the medical device regulation (EU) 2017/745.

## Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.

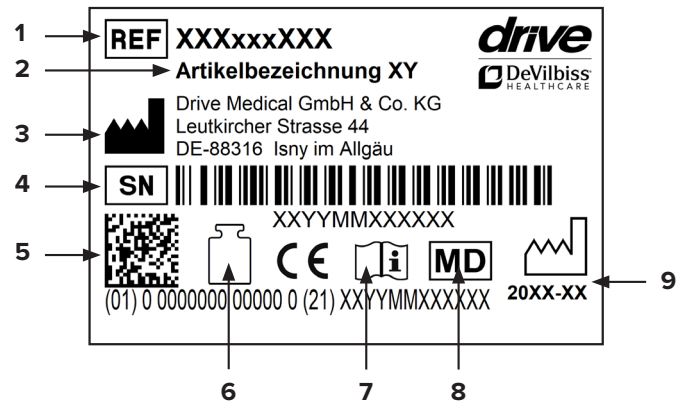
This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee

## Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

## Identification plate

1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year-month)

## Reuse

This product can be reused.

Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available from your specialist dealer or at [www.drivedevilbiss.de](http://www.drivedevilbiss.de)

## Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

### EXCLUSION OF WARRANTY

Parts subject to wear include (among others): wheels, brakes and handles

## Parts of the product

## NITRO GLIDE

- |                       |                       |                                 |
|-----------------------|-----------------------|---------------------------------|
| 1. Handgrip           | 5. Knee Post Lock Tab | 9. Basket Receptacle            |
| 2. Handbrake          | 6. Disc-Brake         | 10. Front Axle Adjustment Knobs |
| 3. Handle height knob | 7. Tiller Locking Tab | 11. Steering wheels             |
| 4. Knee pad           | 8. Basket             | 12. Rear wheels                 |

## Purpose

The walker is for assisting persons with chronic or temporary functional impairments to the foot, ankle and/or distal lower thigh. This walker is a product consisting of a tubular frame and four solid rubber wheels. These walkers are designed such that the user can place the affected extremity on the pad provided for this purpose and relieve the load on it. For improved mobility and patient safety, always use the stronger leg for pushing/operating the knee walker for both indoor/outdoor use. These walkers have braking systems which operate using Bowden cables operated by handgrips. Some of the products feature devices or baskets for holding loads and objects, which can be transported together with the frame. These walkers can be folded up for ease of transport, whilst the handgrips and knee pad are both height-adjustable for resizing.

## Indication

Foot-relieving rollator which is used as an aid for assisting persons with impaired mobility due to injuries on the foot, ankle and/or distal lower thigh. The injured non-weight bearing leg should be placed on the padded knee pad of the walker. Gripping the handlebar with your hands, then using the healthy leg to push off, you can propel the Nitro Glide in a rolling motion whilst the body weight is supported by the knee pad.

## Contraindication

Use of the rollator is not appropriate for:

- Balance disorders
- Impairment to both lower limbs
- Acute thrombosis of the affected leg
- Load restrictions of the knee joint on the affected leg
- Active wound infections on the affected leg
- Serious polyneuropathy with impairment of sensitivity in the lower thigh and knee joint of the affected extremity
- In case of polytrauma with impairment to both the lower and upper extremities

## Scope of delivery

1 x	Knee Walker
1 x	Basket
1 x	Back support
1 x	Instruction for Use

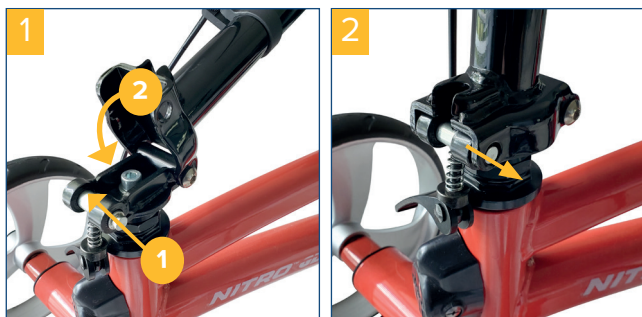
## Safety instructions

- Use the rollator only as a walking aid!
- Use the rollator only on an even and solid surface.
- Rollator may not be used as a seat in vehicles, such as a bus or train!
- It is prohibited to stand on the rollator or to transport a person or animal on the rollator.
- Take note of the permitted maximum weight of the user.
- This information can be found in the technical data in these instructions for use as well as on the identification plate.
- Use the tray only for small, lightweight objects which are not sharp or hot. Max. load: 5.0 kg
- Use the basket only to transport objects and do not overload it! Max. load: 5.0 kg
- Use on escalators and similar structures is expressly prohibited!
- Do not hang any bags on the handle – risk of tipping!
- On steep surfaces, use the service brake.
- Leaning excessively on the handles can cause the rollator to tip forwards.
- When folding/unfolding the rollator, clothing and body parts can be pinched.
- Protect the rollator from direct sunlight. Product parts can become hot and cause skin injuries if touched.
- Prior to each use, ensure that your rollator is intact and all adjustable parts are properly secured. Check that the brakes are working properly!
- Before using the rollator, ensure that the folding guard is properly locked.
- Check the screws on the rollator regularly and tighten, if necessary.


## Assembly

1. Take all parts out of the box


2. Align the handlebars in a vertical position. Push the locking mechanism to the side so that the handlebars can reach their full vertical position (**Fig. 1**). Release the locking mechanism to lock the handlebars into position (**Fig. 2**).

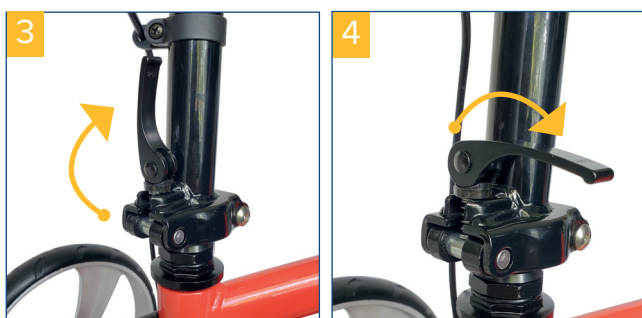3. Push the locking tab down (**Figs. 3 & 4**) to secure the handlebars. If the locking tab feels loose, raise it and turn it anticlockwise to tighten it. Press the locking tab down to secure it.

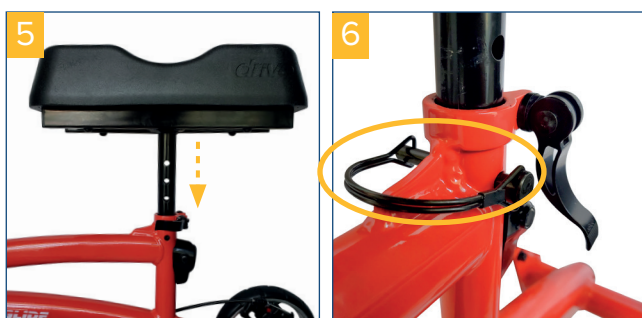4. Guide the knee pad into the holder on the frame (**Fig. 5**) Secure the knee pad by inserting the locking pin in at the desired height (**Fig. 6**).

5. Lock the height of the knee pad by closing the locking tab (**Fig. 7**).


6. Attach the basket by moving the basket holders down using the hooks on the front side of the handlebars (**Fig. 8**).

## Height adjustment
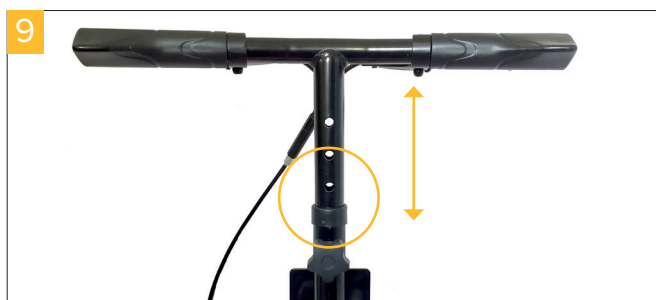
### Push handles

To set the handle height, turn the adjusting knob on the handlebars anticlockwise until it can be removed (**Fig. 9**). Set the handgrips to the desired height. To fix the handlebars into position, screw the adjusting screw back in and tighten it.


### Knee Pad

To set the height of the knee pad, loosen the locking tab below the pad (as shown in step 5 of the assembly guide) and remove the locking pin (as shown in step 4 of the assembly guide). Set the pad to the desired height, re-insert the locking pin and secure it with the locking tab. Tighten the locking tab to fix the pad into place.

## Adjustment of the front axles

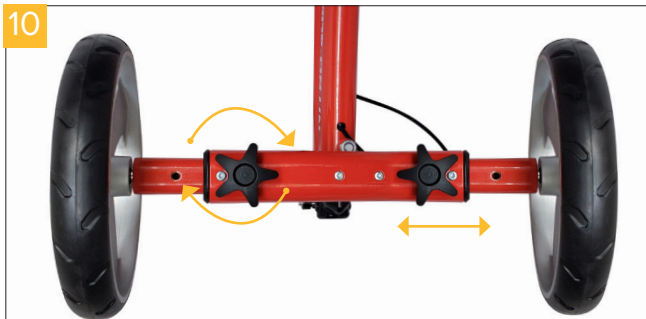The front axles on the foot-relieving rollator can be set to three different widths (40 cm, 45 cm, 50 cm). The product is delivered on the narrowest width setting, whilst the three linear markings on the front axles show the various width options.

To set the width of the front axle, turn the foot-relieving rollator on its side. Turn the adjustment knob on the front axle anticlockwise until it can be removed (**Fig. 10**). Set the position and then re-insert the adjustment knob. Turn this clockwise until it is tight. Repeat this procedure on the other side and pay attention to the linear markings to ensure that both sides are set at the same width.

## Setting brake

If braking is too weak or too strong, then the brake can be adjusted on the brake housing on the handle (**Fig. 11**). Turn anticlockwise to tighten the Bowden cable. Turn clockwise to loosen the Bowden cable.

The brake is set correctly if the brake lever and the brake cables have no play, and the wheels turn without any grinding noises when the brake is released.

## Braking/ Parking

**Important: Do not use the foot-relieving rollator if the braking system does not work properly.**

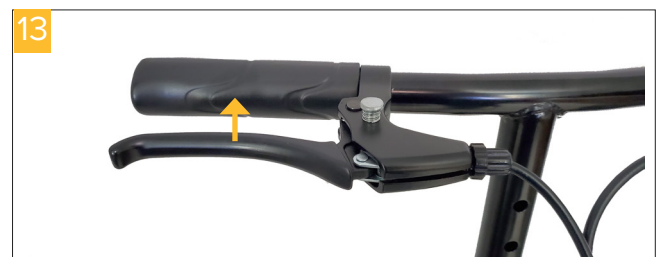
The brake on the foot-relieving rollator is similar to that on a bike and has a locking function.

Simply pull the handlebar with the fingertips on the lever to actuate the brake. The brake works on both wheels simultaneously, regardless of whether you press the left or right brake lever.

Always brake slowly and evenly.

To activate the parking brake, pull the brake lever towards the handlebar and press the spring-loaded button (**Fig. 12**). If handled correctly, the button remains in the pressed position and the braking lever is locked.

To release the brake, simply pull the brake lever towards the handlebar (**Fig. 13**). The button automatically pops back up.

## Fold/ Transport/ Stowing

To fold the rollator, loosen the locking tab on the handlebars (as in step 3 of the assembly guide) and push the locking mechanism to the side (as in step 2 of the assembly guide) until the handlebars are released. Fold the handlebars down until they rest below the knee pad.

## Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- The product should not be exposed to temperatures above 60° C for long periods of time (for example, use in a sauna, etc.)!
- In the event of defects, contact your specialist dealer from whom you purchased this product!
- Check the condition of the tires regularly. They are „fail-safe“, meaning that you do not have to check air pressure or fill them with air. Always check the tires for tears or other damage, however.
- Corrosion-protected materials were used to manufacture the product. If, nevertheless, damage is detected in terms of corrosion, the foot relief rollator must no longer be used and the specialist dealer must be contacted immediately.

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Drive Medical GmbH & Co. KG