TRS 200 XXL

TOILETTENROLLSTUHL / WHEELED DROP-ARM COMMODE

CE

Vorwort

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CE

Foreword

Dear users,
we congratulate on the buying of a technically matured and the best available technology product.

This instructions helps you and the carers as a basis for sure, safe use of the toilet wheel chair and a sensitive use of the product's possibilities.

The instructions describes the preparation, service, use, servicing and cleaning of the product.

This instructions is to be perused carefully and must be always accessible to all users of the product.

Scope of derlivery contents

1. Wheeled Drop-Arm commode
2. Bucket with lid
3. User Manual

Indication

Toilet wheel chairs are necessary when using or achievement of the toilet is not possible because of a handicap. Toilet wheel chairs are also usable as room wheel chairs. Toilet wheel chairs can be pushed over customary toilet bowls, up to a height of 41 cm.

Safety instructions

- Read this instructions carefully.
- Before use all components are to be checked for damages and firm seat.
- While seating, rising and while inserting the toilet bucket as well as folding up or engaging the armrests it`s possible to pinch clothing or body parts
- The use is allowed only on level grounds
- Avoid moisture – Slide danger !
- With use of the toilet wheel chair always operate the full stop brake
- Before getting in / from the toilet wheel chair swing away the footrests
- Footrests never use as stand up device - Tilting danger !
- The steadiness of the toilet wheel chair is clearly lower as with wheel chairs, therefore avoid lateral lean out – Tilting danger !
- Protect from unauthorized use, e.g., by children, by locking the brakes
- The toilet wheel chair TRS 200 XXL serves excluding the use of a person to max. 200 kg on the seat and not as a transport possibility of objects (e.g., to ladders, buckets or similar)!

ATTENTION: While driving reverse across door sills with the loaded commode chair it causes the risk that the interlock of the armrests get opened.

Part of the Wheeled Commode

- | | |
|---------------------------|------------------------------------|
| 1. Push handle | 7. Double function full stop brake |
| 2. Backrest | 8. Armrest |
| 3. Toilet seat | 9. Footrest |
| 4. Fixation clip backrest | 10. Footrest interlock |
| 5. Bucket with lid | 11. Heel tape |
| 6. Armrest interlock | |

Assembly

The commode is supplied in pre-assembled modules. (Picture 1)

The installation of the Wheeled Drop-Arm commode requires no special knowledge and is mountable nearly without tools.

Put the premounted backrest on the basic rack and protect with the fixation clips for backrest. (Picture 2)

Inter- and unlock of armrest

Pull both armrests upwards until they audible engage the interlock.

The interlock occurs automatically. Make sure that the interlock is successful on both sides, on this occasion, the locking pin (1) must be visible. (Picture 3)

In order to unlock the armrests move the locking button to the back (2) and swing the armrest downwards. (Picture 3)

ATTENTION: It may be possible that the armrest is a little bit bent during transport, this can be postbent easily until the locking button in front properly locks.

Double function Full stop brake

To interlock the brake press lever downwards. This will lock the direction of the wheel in place (Picture 4). To unlock press the small lever at the top forward with the shoe (Picture 5).




Fitting footrest

Hang up the provided footrests with both drillings as shown in the respective hold pins. (Picture 6-7)

Interlock and unlock footrest

In order to interlock the footrests turn them to the middle until its audible engage.

In order to unlock press the lever as shown inwards. (Picture 8)

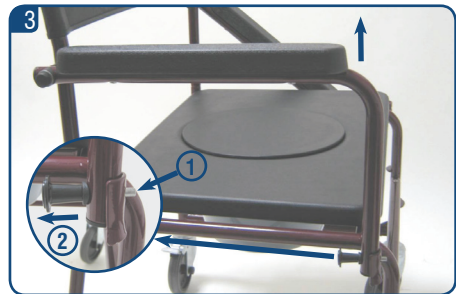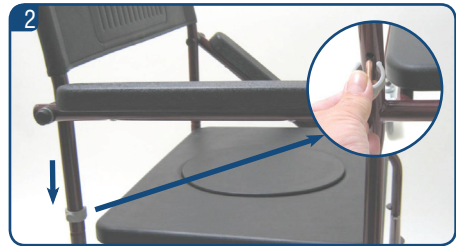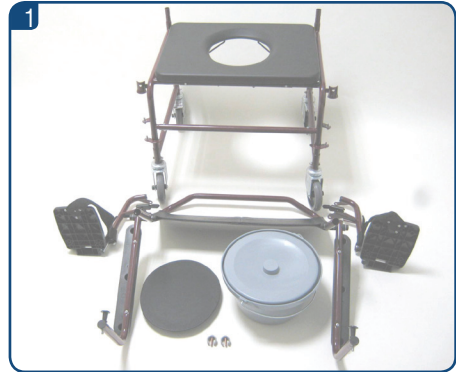Height Adjustment of footrest

To adjust the footrests in the height first unlock the nut (wrench 13mm). Then the footrests can be shifted stepless in the height for leg lengths from 420 – 570 mm. (Picture 9)

ATTENTION: Never use the foot rest as stand up device !

Bucket

Insert or remove the toilet bucket (also with put on lid) always from the top. The toilet bucket engages, the toilet seat can be opened simply upwards for cleaning. (Picture 10)

Cleaning/Disinfection/Maintenance

- Clean regularly by hand with a damp cloth and usual household cleaning agents!
- Use spraying disinfectants p.e. Descosept AF or Bacillol AF.
- Do not use any aggressive detergents!
- Screws of TRS 200 XL must be checked regularly and if necessary retightened.

Technical Data

- 1 Product name
- 2 Article number
- 3 Date of Production and serial number
- 4 Attention: See operating instructions!

Warranty

We issue a warranty of 24 months from date of purchase.
Subject to technical alterations.
Errors and omissions excepted.

Technical Data

Width	72 cm
Length w/o footrest	62 cm
Length w/i footrest	88 cm
Height push handle	98 cm
Height of backrest	42 cm
Angle of backrest	9°
Depth of seat	42 cm
Height of seat	52 cm
Width of seat	56 cm
Adjustment of footrest	42 - 57 cm
Wheels diameter	12,5 cm
Weight	15,3 kg
Max. loads	200 kg
Materials: Frame: Welded steel Tube, powder coated. Toilette seat and armrest: PU (Polyurethane)	

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