Bedienungsanleitung  
Instruction manual  
Manual de instrucciones  
Mode d'emploi

**Pyro Next**


DE

EN

ES

FR

*Enjoy mobility.*

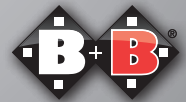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## Index

1.	Preliminary remark	34	5.	Use	50
1.1.	Signs and symbols	34	5.1.	Entering and leaving from the	50
2.	Important safety instructions	35	5.2.	Entering and leaving from the	51
2.1.	General safety instructions	35	5.3.	Climbing stairs and high steps	52
2.2.	Anti-tip wheels (optional)	37	5.4.	Operation	53
2.3.	Use on public roads	37	5.5.	Tip assist	53
2.4.	Purpose	37	6.	Technical data	54
2.5.	Indications for use	38	7.	Transport	56
2.6.	Contraindications	38	7.1.	Transport des Rollstuhls	56
2.7.	Declaration of conformity	38	7.2.	Sitz im Behindertentransportfahrzeug (BTF)	56
2.8.	Responsibility	39	8.	Maintenance tips for users	57
2.9.	Service life	39	8.1.	Cleaning and disinfection	57
3.	Description of product and	40	8.2.	Inspection of the wheelchair by the user	57
3.1.	Inspection on delivery	40	8.2.1.	Brake Inspection	57
3.2.	Type plate and serial number	41	8.2.2.	Checking the tyres	58
3.3.	Contents	42	9.	Maintenance tips for retailers	58
3.4.	Summary	42	10.	Transferring the wheelchair	59
4.	Assembly/Adjustments	43	11.	Storage/shipping	59
4.1.	Folding and unfolding	43	12.	Disposal	59
4.2.	Leg rests	44	13.	Warranty	60
4.3.	Side panels and armrest	45			
4.4.	Brakes	46			
4.4.1.	Setting the parking brake	46			
4.4.2.	Adjusting the brakes	47			
4.5.	Seat height	48			
4.5.1.	Hight of driving wheel	48			
4.5.2.	Hight of steering wheel	48			
4.6.	Wheelbase	49			
4.7.	Quick-release axle	49			

## 1. Preliminary remark

Dear User

You have chosen a high-quality wheelchair from Bischoff & Bischoff. Thank you for your trust in our product

This instruction manual contains all necessary information on operating the versions of the Pyro Next.

The BX 11 is designed for use both indoors and outdoors by active users who rely on a wheelchair.

Before using your new electric wheelchair for the first time, please read and follow the instructions in this manual carefully. The instruction manual is an integral and necessary part of the wheelchair. This is why you should keep it close by and include it when passing on the wheelchair to someone else.

Any repairs and adjustments require special technical training and may therefore only be carried out by authorized B+B dealers.

### 1.1. Signs and symbols

Warning! These are especially safety-relevant instructions. Follow the directions in the manual!

Important! This is especially useful information on a given subject.

## 2. Important safety instructions

### 2.1. General safety instructions

To avoid falls and dangerous situations, you should first practice driving your new wheelchair on level, unobstructed ground. We recommend having an assistant with you.

- The accompanying person should be physically and mentally capable of operating a wheelchair.
- Never use the wheelchair under the influence of alcohol or other drugs.
- Please make sure that all parts have been properly installed before using the wheelchair.
- Never lift the wheelchair by parts that are not fixed firmly (the armrests or legrests).
- Make sure that the screwing axle/floating axle are tightly in the axle holder, otherwise there is risk of injuries.
- Please note that displacing the center of gravity by shifting the body or by adding loads may increase the risk of tipping over the wheelchair to the side or backwards.
- Please seek the help of an assistant person when going over obstacles, steps, etc. Use the tip assist (see overview of wheelchair) to tip the wheelchair appropriately.
- Steps and larger obstacles may only be crossed with the help of at least two assisting persons. For this, use lifting points 1 and 2 (see overview of the wheelchair). If equipment such as ramps, lifts or step climbing aids are available, these should be used.
- When driving on an incline or slope, it is necessary to be aware that a risk of tipping exists even at low angles, depending on the axle position setting. The use of anti-tippers is highly recommended.
- The seat materials are resistant to flammability according to EN 1021-1 and EN 1021-2. Nevertheless, you should exercise the necessary caution when smoking and you should not put a cigarette down on the wheelchair. Avoid running against obstacles (steps, kerbs) without braking or moving down steps.
- Avoid using on uneven, loose surfaces.
- When getting into or out of the wheelchair, apply both parking brakes.

- All road traffic regulations must be observed when driving on public roads.
- The passive lighting (reflectors) of your wheelchair must always be visible to other road users. The wheelchair may not be used for transport of more than one person or any additional load. Make sure to only use it for the intended purpose.
- You must not use the handbrake to stop the wheelchair.
- When braking at fast speeds or on long downward slopes when using the push rims, fingers and hand surfaces become hot. Caution! Danger of burning!
- We recommend leather gloves for outdoor use. These increase grip and protect fingers and hands from dirt and injury at the same time.
- Please note that the upholstery will heat up if directly exposed to sunlight and may cause injuries to your skin when touched. You should therefore cover these parts or protect the wheelchair from exposure to direct sunlight.
- Also note that the frame and pad parts may cool down significantly in winter. If possible, do not park the wheelchair outdoors in cold weather.
- When transferring between wheelchair and bed, always ensure that:
  - both parking brakes are applied
  - the side part on the side for getting in/out is swung backwards.
  - the foot plates (for getting in/out) are folded upwards, swing inwards/outwards or are removed.

## 2.2. Anti-tip wheels (optional)

The anti-tip wheels prevent the wheelchair from tipping backwards. Note that driving with anti-tip wheels on terraced terrain, lift platforms and ramps is only possible to a limited extent. Ensure sufficient clearance above, below and on the sides.

## 2.3. Use on public roads

The wheelchair is designed for indoor and outdoor use. Please keep in mind that you are using public roads and therefore have to observe all road traffic regulations.

Do not endanger other road users through inconsiderate driving, especially on walkways.

## 2.4. Purpose

The wheelchair is designed solely to increase mobility and to transport people with impaired mobility in accordance with indications listed. The maximum user load is 125 kg.

To note:

We are only able to issue a guarantee if the product is used in accordance with the specified conditions and for the specified purpose.

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Inability or severely impaired ability to walk due to

- paralysis
- loss of limb
- defect/deformity of limb
- joint contracture
- joint damage (not in both arms)
- other diseases

## 2.6. Contraindications

Provision of wheelchairs is not suitable for people:

who are unable to sit

- with joint damage/joint contracture of both arms
- with perceptual disorders
- with severe equilibrium disorders
- with reduced and inadequate vision
- with serious cognitive restrictions

## 2.7. Declaration of conformity

B+B, as manufacturer and solely responsible party, hereby declares that the wheelchair BX 11 complies with the requirements of the EC Directive 93/42.

## 2.8. Responsibility

We are only able to provide a warranty if the product is used in accordance with the specified conditions and for the intended purposes. If modifications, repairs or maintenance works are made, please make sure only authorised persons perform these works in accordance to the instructions of B+B.

## 2.9. Service life

If used as intended, and the safety, care and maintenance instructions are observed, the expected service life is up to 5 years. Beyond this time the wheelchair can be used continuously, provided it is in a safe condition.

### 3. Description of product and contents

#### 3.1. Inspection on delivery

All B+B wheelchairs are subject to a proper final inspection in our house and receive the CE mark.

The wheelchair is supplied in a special box. After unpacking save the box if possible. You may need it later for storage or to return the product.


Assembly, adjustment to your body size and first instructions are taken care of by your qualified medical supply dealer.

If delivered by rail or a forwarding agent, the product is to be checked with regard to transportation damage at once in the presence of the shipper.

Check the contents for completeness and make sure nothing is damaged. Should you find any irregularities or damage, please contact our customer service (Tel.: +49 700 6000 7070).

#### 3.2. Type plate and serial number

The type plate (fig. 01) and serial number are located on the bottom of the frame.

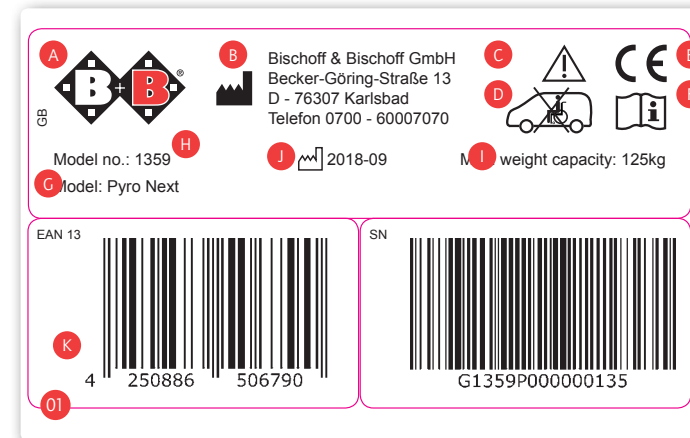- A** Manufacturer logo
- B** Manufacturer information
- C** Caution! - Follow instructions for use
- D** Follow instructions regarding securing the product in the vehicle
- E** CE mark
- F** Important - Adhere to the operating instructions
- G** Model designation
- H** Model number
- I** Maximum weight capacity
- J** Date of manufacture
- K** Serial number

### 3.3. Contents

After receiving the goods, please check the contents for completeness right away. The content consists of:

- Outer package
- Pre-assembled wheelchair
- Instruction manual

### 3.4. Summary

## 4. Assembly/Adjustments

### 4.1. Folding and unfolding

#### Delivery

Your new B+B wheelchair is delivered fully assembled and folded in an original B+B carton.

To prevent damage during transport, attachable mounting parts are delivered separately.

#### Unfolding

Position your wheelchair next to you. Tip it towards you until the load is taken off one driving wheel. Now push the seat tubes apart and press them downward with your palm until they lock in the mounting on the frame (Fig. 03).


#### Folding

Place your wheelchair next to you. Fold the foot plates/the footrest upward (Fig. 4) and undo the Velcro fastener of the back cover under the seat.

Now pull the seat cover upward until the chair is completely folded (Abb. 05).

Make sure that your fingers stay on the frame when folding and setting up the chair. Otherwise there is a risk of entrapment!

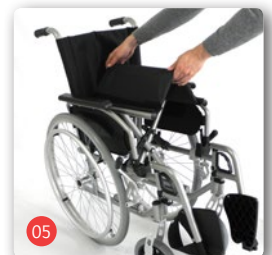## 4.2. Leg rests

The Pyro Next is equipped with leg rests that can be swivelled to the side and removed without tools.

The footplates can be folded upwards and to the side, positioned at an angle and adjusted to length of the user's lower leg.

### Attaching legrests

- Hold the legrests on the upper part, such that this part points outwards.
- Now push the leg rest with the guide profile into the hole in the frame (Fig. 7).
- Swing the leg rests forwards again until you clearly hear them click into place.

### Swinging away and removing the leg rests

To make it as easy as possible to get into and out of your folding wheelchair, you can simply swing the leg rests away, either.

Fold the foot plates up.

- Pull the stop up and fold the leg rest out (Fig. 08).
- In the outer position, the leg rest can be pulled upwards (Fig. 09).

To return to the original position, simply engage the leg rests and pivot them forward until they lock into place.

Before moving off, check for firm attachment of the legrests!

## Setting the legrest length

The foot plates are infinitely adjustable to the length of the lower legs.

- Remove the two Allen socket screws (4 mm) from the leg rest tube (inside and outside), and adjust it to the desired length (Fig. 17).
- Retighten the screws after the adjustment.

When standing up/sitting down on your wheelchair, please avoid standing on the footrest, danger of falling or tipping!

## 4.3. Side panels and armrest

The side panels, which can be swivelled to the back, are also equipped with armrests, which can be fitted in the following three positions: short, standard and long.


### Pivoting the side parts away

- Activate the lever and pivot both side parts backwards and away at the same time (Fig. 11).
- To lock the side part, push it forwards and allow it to click into place with light pressure on the armrest

### Position of the armrests

- Loosen the fixing screws (Allen screw 4 mm) on the bottom (Fig. 12).
- Position the armrest in short, standard or long and tighten the screws again.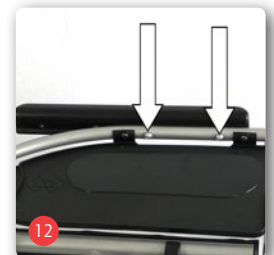

## 4.4. Brakes

The Pyro Next is equipped with parking brakes for user operation as standard.

### 4.4.1. Setting the parking brake

The parking brake should only be used for parking. It is not suitable for braking while driving! Use the push rims on the drive wheels for that purpose. To decelerate, you grasp the highest point of the push rims with your hands and let them be carried along by the wheel (Fig. 30). The push rims initially slide between your hands. Depending on the force you exert when holding the push rims, you can change the braking force from controlled deceleration to a sudden stop.

After the wheelchair has been decelerated in this way, activate both tyre brakes by pushing the brake lever forwards. The wheelchair is now securely braked (Fig. 14).

Pull the lever back towards you to release the brake (Fig. 15).

The parking brake only affects one drive wheel!

### 4.4.2. Adjusting the brakes

If you notice that the available braking effect is no longer sufficient or is uneven, proceed as follows:


- With pneumatic tyres, always ensure that both drive wheels have the same tyre pressure to achieve good directional stability!
- Check the tyre tread. Check also that the friction is even. If it is not, or if the tread is no longer even visible, the tyre must be replaced.
- If you have the necessary technical knowledge, you can change the tyre yourself. If you do not feel confident about doing this yourself, your qualified mobility dealer will be able to help you.
- Now check the adjustment of the brake. This must be set so that the gap between the surface of the wheel and the braking bolt is 8-10 mm with the brake off (Fig. 16). The braking bolt must be aligned horizontally, that is parallel to the ground surface.
- To adjust the gap, loosen the two Allen screws on the brake mounting (Fig 17).
- You can then push the braking system along the mounting into the correct position.
- Then tighten both Allen screws again.
- Make sure that the brake fully locks into place when the lever is operated and does not snap back.
- After completing the adjustment process, tighten the screws again and test the brakes!

Adjustment of the brakes should be carried out by trained specialist staff. However, if you have enough experience, you can also carry out this work yourself. Nevertheless be aware that incorrectly adjusted brakes may have life-threatening consequences.

## 4.5. Seat height

The seat height can be adjusted to your requirements in just a few steps. Driving wheel and steering wheel must be in the same position in each case. TOP – MIDDLE – BOTTOM.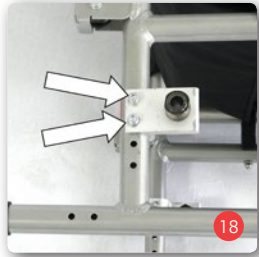

### 4.5.1. Hight of driving wheel

- Take the driving wheels off using the quick-release axle. (Fig. 22).
- Loosen the fixing screws (Allen screw 4 mm) on the bottom (Fig. 12).
- Position the armrest in short, standard or long and tighten the screws again.
- Adapter AT THE TOP means minimum seat height. Adapter AT THE BOTTOM means maximum seat height.
- Insert the screws back into the adapter and tighten the screws.

### 4.5.2. Hight of steering wheel

After changing the height of the driving wheel, the steering wheel has to be put in the same .

- Loosen the axle (Allen key, SW 10) and take it out completely. (Fig 20).
- Place the steering wheel in the right position for the required seat height
- Reinsert the axle into the steering fork and tighten it up again.

## 4.6. Wheelbase

The Pyro Next is supplied with a short wheelbase as standard. Extending the wheelbase increases the stability and equilibrium of the wheelchair.

- Remove the drive wheels (Fig. 22).
- Loosen the screws on the adapter (Allen screw 5 mm and ring spanner SW 10) and pull them out completely (Fig. 18).
- Position the adapter according to Fig. 21, with the seat height adjusted if necessary.
- Reinsert the screws into the adapter and tighten them up again.

When the wheelbase is changed, it is absolutely essential to adjust the knee lever brake afterwards.

## 4.7. Quick-release axle

The Pyro Next comes with quick-release axles as standard. The quick-release axles allow the wheels to be fitted and removed easily.

- Press the locking button of the quick-release axle with your thumb and pull off the wheel (Fig. 22).
- When refitting the wheel, simply push the wheel back onto the mount. At the same time, press in the locking button as well.

When fitting the drive wheels, make sure that the quick-release axle is inserted right up to the stop. After attaching, check that the wheel is securely fixed by pulling it towards you without operating the locking button of the quick-release axle.

## 5. Use

### 5.1. Entering and leaving from the side

- Move the wheelchair as close as possible to the present or future seat so they are side by side.
- If you wish to move from another wheelchair or room chair, lock it by setting the parking brakes.
- Secure the Pyro Next against rolling away accidentally by activating the parking brake.
- Pivot the armrest backwards and away.
- Fold the foot plates to the side or swivel the leg rests to the side
- Now slide sideways onto the other seat.
- Make sure that you are sitting as far back in the seat as possible.
- Finally, return the leg rest to their original position and reinsert the side parts if necessary.

Never stand on the footrest!  
Risk of tipping!

### 5.2. Entering and leaving from the front

- Swivel the foot plates upwards or swivel the leg rests to the side.
- Move the Pyro Next as close as possible to the current or intended seat so that it is facing the seat and apply the brakes on the Pyro Next.
- If you want to switch from another wheelchair or indoor frame, secure it by activating the locking brakes.
- Now slide onto the seat by turning your body.
- Finally, return the leg rests to their original position.

Never stand on the footrest!  
Risk of tipping!

### 5.3. Climbing stairs and high steps

Where steps are high and where there are stairs with more than 3 steps, 2 assistants must help.

- The helpers must only take hold of securely fitted parts of the frame. Please see Fig 23.
- When going up stairs, the helper behind the wheelchair pulls the wheelchair over the step holding it by the securely fitted handles.
- The helper who is below the chair takes hold of the front frame tubes and stabilises the position. At the same time, this helper pushes the rear wheels of the wheelchair against the steps.
- When going down stairs, the helper who is below the chair brakes by pushing the wheelchair right against the steps. The helper who is above the wheelchair holds it by the push handles, secures it and holds it in the right position. The wheelchair should roll down step by step.

The frame and the push handles are the only places where the wheelchair can be gripped to transport it. Under no circumstances should armrests be used to lift the wheelchair!

### 5.4. Operation

Sit in your B+B wheelchair. Make sure you are sitting as far back as possible. The push rims on the drive wheels are used to move the wheelchair. Your hands should be positioned with the thumbs and curled index fingers on the push rims. From this position, push the wheelchair with both arms equally.

To drive to the right, hold the right push rim in place and only push the left one. To drive to the left, perform the same pushing motion the other way around.

To turn in place, push with both hands in opposite directions at the same time.

To decelerate, reduce the speed of the push rim with your hands (see also section 4.4, „Brakes“).

Practice moving the wheelchair carefully at first until you are familiar with it and the way it behaves. Wheelchairs have only a limited ability to resist tipping and sliding. Particular care is necessary when braking and moving onto, or turning on inclines. Leaning a long way out of the wheelchair increases the danger of tipping.

To overcome obstacles more easily, the wheelchair must be tipped by the accompanying person.

### 5.5. Tip assist

The tip assist can be used to make it easier for an accompanying person to negotiate obstacles such as kerbs. Hold the wheelchair firmly by the push handles with both hands. Then press one of the tip assists with one foot (Fig. 25), while at the same time pushing downwards with your hands.

## 6. Technical data

### **Dimensions and weights**

Total length with/without leg rest (cm):	103/83
Width (cm):	Sitzbreite + 20cm
Height (cm):	95
Folded (BxHxL) (cm):	35x95x103
Height of handle (cm):	94
Lower leg length (cm):	40-48
Footrest angle (°):	5
Knee angle (°):	105
Unloaded weight (kg):	13,4 (SB 40cm)
Max. weight capacity Max. weight capacity (kg):	125
Seat angle (°):	3
Seat width (cm):	40/43/46/49
Seat depth (cm):	42,5
Front seat height (cm):	47/49/51
Rear seat height (cm):	44/46/49
Seat back angle (°):	7 (with lumbar curve)
Seat back height (cm):	43
Height of the armrests (cm):	22
Length of the armrests (cm):	32
Width of the armrests (cm):	4

### **Tyres**

Drive wheels (inch):	24x1 <sup>3</sup> / <sub>8</sub>
Swivelling wheels (inch):	8x1 <sup>1</sup> / <sub>4</sub>
camber angle (°):	0

### **Equipment**

Frame:	Aluminium
Seat and back pads::	Nylon
Push handles / Armrest:	PVC/PU

## 7. Transport

### 7.1. Transport the wheelchair

Wheelchair suitable for use as a seat in a vehicle for transporting disabled people.

To transport the wheelchair as convenient as possible, all detachable parts (footrests, wheels, accessories) must be removed and the wheelchair folded.


### 7.2. Seat in disabled vehicle

You can determine whether or not your wheelchair is approved for use as a seat in a vehicle for transporting disabled people by referring to the symbols on the CE sticker on the product:


All products approved by B+B for use as seats in vehicles are tested in accordance with ISO 7176-19. The tests in accordance with ISO 7176-19 were performed using restraint systems from AMF-Bruns. The correct mounting of the restraint system is a prerequisite for the safe transportation of the products in a vehicle for transporting disabled people and the basis for the approval.

The restraint systems from AMF-Bruns provide the ideal points for the provision of restraint force for the purpose of securing the products in the vehicle for transporting disabled people (known by the German term „Kraftknoten“, which is defined in the standard DIN 75078-2). For the purpose of providing restraint in the vehicle for transporting disabled people (fig. C), the products must only be secured at these specific points („Kraftknoten“ points). If possible, use a permanently installed car seat and its belt system. Stow the wheelchair in the luggage space. The person responsible for the transportation should be familiar with the current regulations for transportation as well as with the method of operating the vehicle for transporting disabled people and the restraint system.

Anchored 4-point restraint systems must be used to secure the wheelchair in the vehicle. If you have any queries about securing your product in a vehicle for transporting disabled people, please contact the B+B customer service department on +49 (0)700/60007070.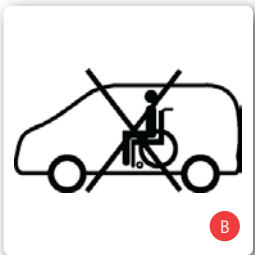

Wheelchair not suitable for use as a seat in a vehicle for transporting disabled people

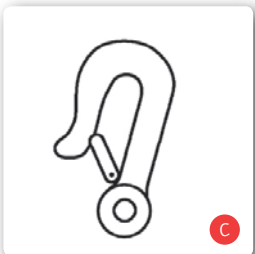Fixation points on the restraint system

## 8. Maintenance tips for users

### 8.1. Cleaning and disinfection

- The frame and the paneling of the wheelchair can be cleaned with a moist cloth. You can use a mild detergent for more stubborn dirt.
- The wheels can be cleaned with a wet synthetic fiber brush (do not use a wire brush!)
- Push handles, arm pads, and seat and back pads can be washed off. Use mild detergent.
- To disinfect the chair, add a commercially available disinfectant, which reliably kills bacteria and germs, to washing water

A list of the disinfectants approved by the Robert Koch Institute can be found at [www.rki.de](http://www.rki.de)

### 8.2. Inspection of the wheelchair by the user

#### 8.2.1. Brake Inspection

Make sure your brakes are operating properly before each use. It should not be possible to push the wheelchair away while the brakes are activated.

The wheelchair should be easy to drive without grinding noises and with good direction stability when the brakes are released.

If you notice unusual behaviour when braking, inform your qualified mobility dealer immediately and do not use the wheelchair.

### 8.2.2. Checking the tyres

Check whether the tyres are intact and inflated to the correct pressure every time before driving.

## 9. Maintenance tips for retailers

We recommend having the wheelchair inspected by your qualified mobility dealer at least once a year. If faults or defects at the wheelchair are found, it must be returned to the dealer for repair immediately.

The dealer can receive the required information and documentation for repair and service on request. A training program is also available for dealers.

B+B customer service is available at +49 (0)700/60007070

The wheelchair must be inspected in accordance to the service plan and must be in good condition

## 10. Transferring the wheelchair

The wheelchair is suitable for continued use by others. Before it is handed on to another user, it should be serviced by a specialist dealer and undergo hygienic treatment. When passing on the wheelchair, please remember to give the new user these operating instructions and the address of your specialist dealer.

## 11. Storage/shipping

If the wheelchair is to be stored or shipped, all attachable and unsecured parts, apart from the drive wheels, must be removed and packed individually in appropriate cartons. The individually packed parts can then be packaged together in a larger carton. We recommend to keep the original carton and store it for those purposes so that it is available if required. Your wheelchair will have an optimum protection against environmental influences during storage and transport.

## 12. Disposal

If the wheelchair is no longer used and you wish to dispose it, please contact your dealer.

If you wish to dispose the wheelchair yourself, please contact a local recycling company for the disposal regulations of your local jurisdiction.

### 13. Warranty

The warranty covers all defects of the wheelchair that are demonstrably due to material or manufacturing defects. Any claim must be accompanied by the completely filled-out warranty certificate.

Warning! Non-observance of the instruction manual, improper maintenance and in particular any technical changes and supplements (added fittings) without the approval of B+B will generally lead to forfeiture of warranty and product liability claims.

Our products comes with a statutory guarantee of 2 years. For replacement devices and repairs, a warranty period of 1 year applies. Please also observe the B+B care, warranty, hygiene and maintenance instructions. Your qualified mobility dealer will be happy to provide these to you.

### Warranty-Certificate

(Please note our general business terms and conditions))

**Model: Pyro Next**

Serial Number: \*

\* (to be filled in by authorized dealer)

**Dealer:**

**Date and Stamp**

\*The serial number is located on the side frame.

#### Tip!

In spite of careful research and editing, incorrect information may have crept into our operating manuals. The current version of each of the operating manuals can be found in the download area of our website: [www.bischoff-bischoff.com](http://www.bischoff-bischoff.com)

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### **Bischoff & Bischoff GmbH**
### **B+B France S.A.R.L.**