Bedienungsanleitung
Instruction manual
Manual de instrucciones

Centuro S2 /S1


DE

EN

ES

Enjoy mobility.

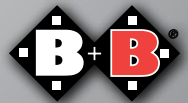Inhalt

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www.bischoff-bischoff.com

Contents

1.	Introduction	46		
1.1.	Signs and symbols	46		
2.	Important safety instructions	47		
2.1.	General safety instructions	47		
2.2.	Information on EMC interference	49		
2.3.	Safety in vehicles	50		
2.4.	Use on Public Roads	50		
2.4.1.	Taking on public transport	50		
2.5.	Brakes	51		
2.6.	Declaration of conformity	51		
2.7.	Information about range	52		
3.	Purpose	53		
3.1.	Indications	53		
3.2.	Contraindications	53		
4.	Description of Product and Contents	54		
4.1.	Inspection on Delivery	54		
4.2.	Type plate and serial number	55		
4.3.	Contents	56		
4.4.	Summary	56		
5.	Basic adjustments	57		
5.4.1.	Steering column	57		
5.4.2.	Seat	57		
5.4.3.	Backrest	58		
5.4.4.	Armrests	59		
5.4.5.	Seat belt (optional)	59		
6.	Operating the scooter	61		
6.1.	Control Functions	61		
6.2.	Getting in and out	63		
6.3.	Driving the scooter	64		
6.4.	Check before Use	64		
6.4.1.	Brake Inspection	64		
6.4.2.	Checking the lights, indicators and horn	64		
6.4.3.	Inspection of Tyres	65		
6.5.	Driving the scooter	65		
6.6.	Pushing mode	67		
6.6.1.	Brakes in pushing mode	68		
6.7.	Overload switch	68		
6.8.	Preparing the scooter for transportation	70		
6.9.	Disassembling for storage	72		
6.10.	Accessories	73		
6.10.1.	Rear-view mirror right (optional)	73		
6.10.2.	Cane holder for 2 walking sticks (optional)	73		
6.10.3.	Licence plate holder (optional)	73		
6.10.4.	Rain cover (optional)	73		
7.	Technical data	75		
8.	Care instructions for the user	77		
8.1.	Cleaning and disinfection	77		
8.2.	Batteries	79		
8.2.1.	Charging	79		
8.2.2.	Replacing the batteries	81		
8.3.	Fuses	79		
9.	Maintenance instructions	79		
9.1.	Maintenance instructions for the Specialist trade	79		
10.	Repair	79		
11.	Storage / Shipping	79		
12.	Disposal	79		
13.	Use by others	79		
14.	Service life	79		
15.	Warranty	79		

1. Introduction

Dear user,

You have decided on a high-quality Bischoff & Bischoff scooter, and we would like to thank you for your trust.

The scooter was designed to offer you many advantages and meet your needs. Thanks to the various adjustment options and optimum adaptability, the scooter can be adjusted according to your individual body size. The scooter is designed for outdoor use only.

Please read and follow the operating manual carefully before using your new scooter for the first time. The operating manual is an integral and necessary component of the scooter. Keep the operating manual where you can refer to it and if you transfer the scooter, give the manual to the new owner.

The scooter complies with the requirements of ISO 7176-14: 2008 (drive and controller systems for scooters and scooters; requirements and testing). For visually impaired users, this document is available as a PDF file at our website www.bischoff-bischoff.com.

Any repairs and adjustments require special technical training and may therefore only be carried out by authorized Bischoff & Bischoff dealers.

1.1. Signs and symbols

Important! This is especially useful information on a given subject.

Warning! These are especially safety-relevant instructions. Follow the directions in the manual!

2. Important safety instructions

2.1. General safety instructions

- To avoid falls and dangerous situations, you should first practice driving your new scooter on level, unobstructed ground. We recommend having an assistant with you.
- Before operating the scooter, please check that all attached parts are correctly fastened.
- Practice driving on level, unobstructed ground before first use. Get a good understanding for the way the scooter brakes and accelerates when driving straight ahead or a curve.
- Please note that displacing the center of gravity by shifting the body or by adding loads may increase the risk of tipping over the scooter to the side or backwards.
- The scooter may not be used for transport of more than one person or any additional load. Make sure to only use it for the intended purpose.
- Be aware of the risk of trapping body parts when making settings and adjustments to the scooter.
- Never use the scooter under the influence of alcohol or other substances that may impair your attention and/or your physical and mental capacity.
- To overcome obstacles, please be sure to use ramps. Should it be necessary to lift the scooter over an obstacle, then only do so without the occupant. Only lift by the solid parts of the frame.
- Avoid hitting obstacles (steps, curbs) at full speed.
- The control unit must always be switched off when getting in or out of the scooter.
- All road traffic regulations must be observed when driving on public roads.
- The rear lights at the scooter must always be visible to other road users.
- If possible, wear bright, conspicuous clothing. This will make it easier for other road users to see you.

- Please note that the upholstery will heat up if directly exposed to sunlight and may cause injuries to your skin when touched. You should therefore cover these parts or protect the scooter from exposure to direct sunlight.
- The seat materials comply with the flammability resistance requirements of ISO 8191-2. Nevertheless, you should exercise the necessary caution if you are smoking and refrain from putting any cigarettes down on the scooter.
- Reporting incidents:

If you, as the operator, user or their relatives, detect possible defects or functional limitations, please contact an authorised dealer immediately. Operators, users or their relatives should inform the dealer who supplied the dealer who provided you with the product of any suspected serious incidents which directly or indirectly have had or could have one of the following and by which they are affected. You can also report this to the competent higher federal authority report

the death of a patient, user or other person,

- the death of a patient, user or other person,
- the temporary or permanent serious deterioration of the health, of the state of health of a patient, user or other persons,
- a serious risk to public health.

2.2. Information on EMC interference

Despite compliance with all applicable EMC directives and standards, it is possible that the scooter may affect or be affected by other electrical devices (e.g. mobile phones, alarm systems in department stores).

If you become aware of any such interference, either turn off the other device or turn off your scooter and move it outside the range of the interference.

2.3. Safety in vehicles

The Centuro S2 is not suitable for use as a car seat. Under no circumstances may it be used as such. It does not meet the requirements stipulated by ISO 7176-19. Failure to observe this instruction can lead to extremely serious or fatal injuries in the event of an accident.

Your scooter is suitable for use on the designated buses in the public transport network (ÖPNV). Please note that the total weight of 300 kg must not be exceeded. The maximum total length of 1200 mm must also not be exceeded by attachments or similar items. Further information on this can be found in Section 4.8

2.4. Use on Public Roads

The scooter is designed for outdoor use. Please keep in mind that you are using public roads and therefore have to observe all road traffic regulations.

Do not endanger other road users through inconsiderate driving, especially on walkways.

2.4.1. Taking on public transport

According to a decree from 2017, your scooter is suitable to be taken on public transport (ÖPNV) and is authorised to bear the corresponding sticker (Fig. B). Please note that the total weight of 300 kg must not be exceeded. The maximum total length of 1200 mm must also not be exceeded by attachments or similar items.

To take your scooter onto a bus, you will need to navigate a ramp in reverse. This takes a certain degree of skill. Practise reversing in a safe environment before your first bus journey.

Not all buses are suitable for electric scooters; please check with the local transport provider in advance.

2.5. Brakes

Service brake:

In order to brake in pushing mode with decoupled drives or in an emergency, a service brake is provided (Fig. 01).

Parking brake:

Switch off the scooter at standstill using the on/off switch (see chapter 5.). If the drives are engaged, the scooter cannot be pushed or moved, as the motors brake automatically (see chapter 5.4).

The function of the service and parking brake complies with the provisions of the German Road Traffic Licensing Regulations (StVZO).

Pushing mode:

A service brake is provided to brake in pushing mode when the drives are decoupled or in an emergency (Fig. 1). The scooter is braked when the drive is engaged by the automatic brake after the drive lever is released. If the drives are engaged, the scooter cannot be pushed or moved as the motors brake automatically.

Please note that operation in pushing mode by disconnecting the drives must only take place on a level surface. The electro-mechanical brake is thereby disabled. There is a danger of fatal injury if this instruction is not complied with.

2.6. Declaration of conformity

We, Bischoff & Bischoff GmbH, declare under our sole responsibility that the Centuro S1 and Centuro S2 scooters comply with all applicable requirements of MDR 2017/745.

2.7. Information about range

The range of the scooter Centuro S1/S2 is 35 km. The range that can be achieved by electric scooters depends on the following factors:

- Composition of the carriageway
- Way of driving
- Battery condition
- Vehicle load weight
- Ambient temperature
- The use of accessories

Please note that the range specified by us was determined under test conditions:

- Batteries as new and fully charged
- Ambient temperature of 21 °C
- max. user weight of 160 kg
- Level, firm surface
- Constant speed
- Deactivated lighting system

We advise you that the following factors severely restrict the range:

- Poor state of charge of batteries
- Low ambient temperature (below 10°C)
- Frequent starting and braking
- Driving uphill
- Driving on surfaces that are not firm (field or forest paths)
- Tyre pressure incorrect or too low

If there is a combination of several factors, the range can be reduced by up to 50 %.

3. Purpose

The scooter Centuro S1/S2 is designed for outdoor use by adult persons with impaired mobility. It is therefore capable of travelling long distances and overcoming obstacles outdoors. The scooter Centuro S2 is classified according to EN 12184 in category B. The maximum load capacity is 160 kg.

3.1. Indications

Inability or severely impaired ability of adults to walk due to

- paralysis
- loss of limb
- defect/deformity of limb
- joint contracture
- joint damage (not in both arms)
- other diseases

Provision of a scooter is advisable if the use of manually operated scooters is no longer possible because of the disability but the person can still competently operate a vehicle driven by an electric motor.

3.2. Contraindications

The provision of a scooter is unsuitable for adult persons

- with severe balance disorders
- with impaired or insufficient vision
- with strongly impaired cognitive abilities

4. Description of Product and Contents

4.1. Inspection on Delivery

All Bischoff & Bischoff products undergo a proper final inspection in our company and are labeled with the CE mark.

The scooter is delivered in a special carton. Keep the carton after unpacking, if possible. It is useful for later storage of the product or for sending it back.


Delivery and first instructions are taken care of by your qualified medical supply dealer.

If the scooter is shipped by rail or road, the goods should be checked immediately for transport damages (carton) in the presence of the bearer.

Check the contents for completeness and make sure nothing is damaged. Should you find any irregularities or damage, please contact our customer service.

4.2. Type plate and serial number

The type plate (Fig. 02) and serial number are located on the back underneath the battery cover.

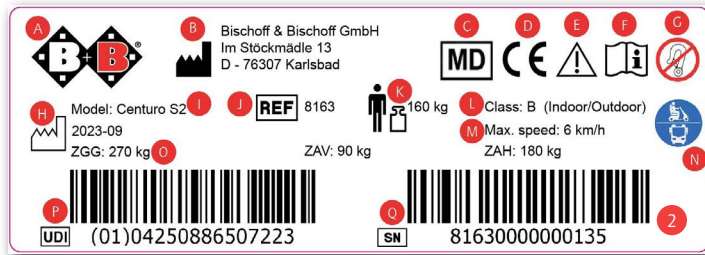- (A) Manufacturer logo
- (B) Manufacturer information
- (C) Medical device
- (D) CE mark
- (E) Caution! - Follow instructions for use
- (F) Important - Follow instructions for use
- (G) Not suitable for use as a seat in a vehicle
- (H) Date of manufacture
- (I) Model number
- (J) Model designation
- (K) Maximum user weight capacity
- (L) Application class
- (M) Maximum speed
- (N) Suitable for use on designated buses in the public transport network
- (O) Permitted loads: Total/front/rear
- (P) UDI number
- (Q) Serial number

4.3. Contents

After receipt of the goods, please check the contents for completeness right away. The contents consist of:

- outer package
- Scooter (ready for use incl. 2 batteries)
- Charger
- Operating manual
- Seat
- Armrest
- Headrest
- Basket

4.4. Summary


5. Basic adjustments

Before using the scooter for the first time, it is necessary to adjust the scooter according to your height. This is essential to ensure that your posture is physiologically correct for driving for extended periods of time without excessive fatigue. The basic adjustments of the scooter are described below. You should carry out these adjustments with the help of your medical supply dealer.

5.4.1. Steering column

Adjusting the steering column of your scooter enables you to set the driving position so that it is comfortable. You can make the adjustment while seated in the driver's seat.

- Hold the handlebar with one hand and activate the adjusting lever with the other hand (Fig. 04).
- Move the steering column into the required position and release the lever.

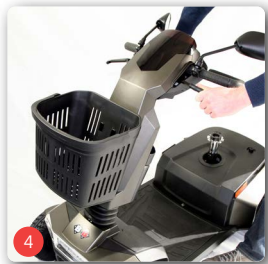5.4.2. Seat

The seat height can be adjusted and the seat can also be moved forwards and backwards as well as rotated to the side.

Rotating the seat

- Push down the lever (located on the right underneath the seat surface, Fig. 5). The seat can now be rotated in both directions.
- There is a fixed lock-in position for the seat every 45°.
- Release the lever and make sure that the seat is locked in position again.

Moving the seat

- Pull the lever (on the right below the seat surface, Fig. 06) towards you. The seat can now be moved forwards and backwards.
- You can now adjust the seat to the required position.
- Release the lever and ensure that the seat is locked in one of the closest fixed positions.

Seat height

- You will firstly need to remove the seat.
- Fold the back rest onto the seat by holding the back rest with your right hand and using your left hand to lift up the lever on the left next to the seat surface (Fig. 09). The back rest will then fold forwards automatically.
- Pull the lever upwards and simply remove the seat by pulling it upwards (Fig. 6/7).
- Undo the screw connection (Fig. 8) and position the seatpost in the required position inside the tube guide.
- Then securely tighten the screw connection again (Fig. 8) in the next hole.
- Now place the seat back onto the mounting.
- Make sure that it is locked in position again.

5.4.3. Backrest

The angle of the back rest on your scooter can be adjusted forwards and backwards.. You also have the possibility to limit the adjustment range backwards.

Adjusting the angle

- Pull the lever (on the left on the seat, Fig. 09) upwards. The back now pushes forwards.
- Push the back rest into the required position.
- Release the lever and make sure that the back rest is locked in position again.

5.4.4. Armrests

The arm rests can be swivelled upwards in order to make it easier to mount and dismount. In addition, the height, distance and angle of support of the arm rests can be adjusted.


Height

- Undo the hand screw until you can pull it out, and hold the screw in position. The armrest can now be adjusted upwards or downwards (Fig. 10).
- Move the armrest to the required height and tighten the screw again.

Side distance

- Undo the hand screw (Fig. 11) on the back of the back rest frame while holding onto the armrest.
- Adjust to the required distance and tighten the screw again.
- The armrests can also be removed (e.g. for replacement, storage or transportation) after undoing the screw on the back of the frame (Fig. 11).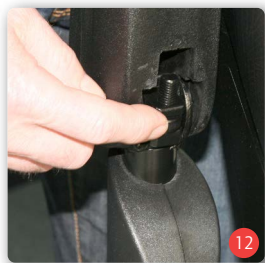

Angle of support

- You can adjust the angle of support of the armrests by screwing the knurled nut (Fig. 12) on the relevant armrest further in or by unscrewing it.

5.4.5. Seat belt (optional)

The seat belt will be attached to the frame from the side.

- Adjust the belt length to meet your needs by positioning the clasp accordingly.
- Fasten the belt with the belt buckle.
- Unfasten the belt by pressing the red button (Fig. 12) on the belt buckle.

6. Operating the scooter

After you have adjusted the scooter according to your height, familiarise yourself with the functions of the controller.

6.1. Control Functions

The controller is a component of the steering column of your scooter. It contains LED displays and functional elements.

Hazard warning lights

Use this button (Fig. 15) to switch on the hazard warning lights in emergency situations where it is not possible to drive the scooter. All four indicator lights then flash and signal to other road users that your scooter cannot be driven.

Speed setting

You can use these buttons (Fig. 16) to set the maximum speed.

- Turn the switch to the left (turtle) to reduce the maximum allowed speed.
- Turn the switch to the right (rabbit), to raise up to the maximum allowed speed.


Headlight

Use this button (Fig. 17) to switch on the lights on your scooter when dusk begins to fall or when passing through dark routes such as tunnels. This will make you more easily visible to other road users.

Diagnostic LED

If the diagnostic LED is constantly lit (Fig. 18) the scooter is fully functional.

If a fault occurs, this is indicated by flashing. First check the battery status and whether the drives are connected. If the LED continues to flash, contact your mobility dealer.

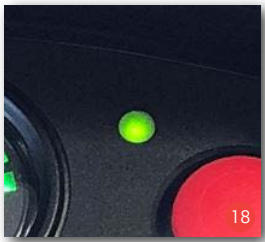Battery Status Display

The battery status display (Fig. 19) shows to what extent the batteries are charged. If all ten bars are visible, the battery is fully charged.

As you use the scooter, the battery capacity decreases and fewer bars are displayed. If only two bars are displayed, this means that only short journeys can now be made and the battery should be recharged (see charging).

Indicators (left/right)

Use this button (Fig. 20) to switch on the left or right indicator on your scooter to signal to other road users that you wish to turn left/right. Pressing the button again turns the indicator off again. After flashing 15 times the signal turns off automatically.

Horn

The horn (Fig. 21) enables you to draw attention to yourself in difficult situations.

Ignition key

Switch the scooter off and on using the ignition key (fig. 22).

6.2. Getting in and out

Ensure that the scooter is switched off before sitting on it.

- Move the steering column, as described previously, into the upright position and swivel back the arm rest on the side from which you intend to sit down so that the arm rest is out of the way.
- It is easier for you to sit down if you rotate the seat 90° to the side from which you intend to sit down (see the section entitled Basic adjustments).
- Now sit down.
- Rotate the seat in the driving direction or swivel the arm rest forwards again (chap. 5.).

 Ensure that the seat firmly locks in position again.

- Adjust the steering column so that you can comfortably reach all the controls.
- To get out, follow the instructions accordingly in reverse order. However, take out the key beforehand so that you can ensure that no unintended actions or movements occur.

6.3. Driving the scooter

6.4. Check before Use

6.4.1. Brake Inspection

Make sure your brakes are operating properly before each use.

- Start off carefully and let go of the throttle lever. The scooter must come to a complete stop as usual.
- When stopped, the mechanical brakes must engage with an audible click. Should you notice any unusual braking behavior, please notify your dealer right away and ensure the scooter is not being used.
- The motor brake is functioning correctly if you can't push the scooter when it is turned off.
- Should you nevertheless be able to do so, then the electro-magnetic brakes are defective. In this case you may no longer use the scooter. Contact your dealer immediately.

6.4.2. Checking the lights, indicators and horn

Make sure your lights, indicators and horn are functioning properly before each use.

- If a unit under inspection does not react when the corresponding button on the control panel is pushed, please notify your dealer immediately and ensure the scooter is not being used.

6.4.3. Inspection of Tyres

Before embarking on any journey, check that the tyres are in good condition.

- Check the tyre pressure. This should be 3.5 bar.

- Carry out a regular check of the tread depth of the tyres once a month. This should be at least 1 mm.
- Any faulty or worn out tyres must be changed through your mobility dealer.

6.5. Driving the scooter

You should make your first journey on a level site with ample space.

- Ensure that you are sitting correctly on your scooter. Set the speed as low as possible. You can increase the speed when you are more familiar with handling the scooter.
- To switch on your scooter, insert the ignition key into the ignition lock on the control panel and turn the key a quarter turn to the right (fig. 22).
- To switch off the scooter, turn the key a quarter turn to the left. After getting out of the scooter, always ensure that you have removed the ignition key and do not leave the scooter unattended with the ignition key inserted.
- Please make sure that you do not have your hand on the forwards / backwards lever when you switch on your scooter. This would cause your scooter to display an error message. In this case, turn it off and then on again to rectify this error.
- Pull back the speed selection lever on the right to move forwards. The further you pull back the lever, the higher the speed. The speed can go up to the maximum speed that has been preset using the speed control. Release the lever to stop gently. Practice these two procedures in order to get used to them (fig. 23).
- Steering is easy and works exactly like a bicycle in that you turn the handlebars in the direction in which you wish to move. Leave plenty of distance when you travel around a curve so that the rear wheels are able to follow the path of the front wheels and so that you do not cut across objects.
- It is necessary to pay attention when reversing. Ensure that your speed is low before reversing. Please keep in mind that you need to steer in the opposite direction when you are reversing. This requires practice. Practice this also on a level site with ample space.

- Pull back the left speed selector lever (Fig. 24) to move backwards. The further you pull the lever back, the higher the speed. Release the lever again to stop gently. Practise these two operations to get used to them.

Please note that, for safety reasons, the reversing speed is only half the speed going forward.

- If you turn sharply, turn the handlebars before you accelerate. Always drive in a wide arc around corners and obstacles and ensure that you drive slowly and carefully. For your own safety, the scooter automatically reduces the speed when cornering.
- When the drive is engaged, the brake is automatically applied when the throttle lever is released. The scooter then cannot be pushed or displaced because the brake is automatically applied to the motor.

6.6. Pushing mode

If you want to push the scooter in certain situations, you must unlock the brakes. It is easy to push the scooter in free wheel mode.

The electro-mechanical brake will then not operate.

- Pull the unlocking lever in the back (Fig 25) downwards.
- A small jerk will indicate that the drives are now unlocked.
- To lock, simply push the lever up and locking then takes place automatically.
- The motor has a centrifugal clutch; if you accelerate the scooter too much, the brake locks automatically. It is released again after the scooter has come to a standstill.

For safety reasons, the motor drive is locked when the scooter is being operated in pushing mode.


Caution! Danger of fatal injury! The drives must only be disconnected on a level surface.

6.6.1. Brakes in pushing mode

A manual service brake is attached to your scooter (as described in section 2.5). This can be used for support with braking when the drives are connected; it can also be used as a service brake when the drive is disconnected. The service brake acts on both front wheels and also serves as a locking brake.

Parking and locking the brake is performed by locking the drive. For this purpose, the scooter must be brought to a standstill by a helper and/or by using the manual service brake. The electro-mechanical brake, as shown in figure 26/27, must then be activated.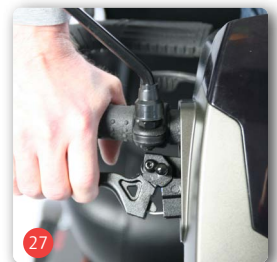


6.7. Overload switch

The controller will cut out if the motor is overloaded, e.g. by driving uphill for too long. In this case, the ignition just needs to be switched off and on again in order to start moving again. If this does not work, this means that the controller has overheated.

In this case, please wait a few minutes. In the event of an electrical overload, the fuse will shut off the power to the scooter. The overload switch (Fig. 28) is located on the front of the seat construction (Fig. 29). After pressing the switch, the scooter should function normally again.

If it does not function as expected, please contact your specialist retailer or Bischoff & Bischoff customer service.

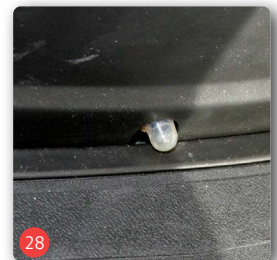6.8. Preparing the scooter for transportation

- You can fold up the scooter in order to save space during transportation.
- Switch off the scooter.
- -Remove the armrests (Fig. 30) and then fold the back rest forwards onto the seat as described in Section 5.
- Remove the basket (Fig. 31)
- Fold the steering column towards the foot area (Fig. 32).
- Depending on the available space, it is sufficient to fold down the back rest and steering column and secure the scooter using securing straps on the frame.
- Carefully protect the scooter against slipping by attaching the securing straps at the designated holding points (Fig. 34, I, II).
- The scooter, including the batteries, is generally suitable for transportation on an aircraft. You can find out the transportation conditions from your airline. The batteries may need to be removed. Information on this can be found in Section 6.9.

6.9. Disassembling for storage

For storage, you must also remove the batteries, be extremely careful. Avoid handling metal tools, touching the battery terminals and observe the instructions of the battery manufacturer.

This work should only be done by a trained technician. There is danger of fatal injury!



For your safety, the seat must be lifted by two people, as it is the heaviest individual part (17 kg).

- You will firstly need to remove the seat and the panel.
- Remove the seat by pulling it upwards.
- The panel is fixed in place with 2 hand screws.
- Undo these screws and remove the panel.
- Disconnect the plug connection for the rear lighting.
- The batteries should then be removed. To make this easier, the cables are fitted with plug connections.
- Disconnect the plug connections (Fig. 36) for both batteries.
- Then undo the screw on the mounting bracket (Fig. 37). You do not need to completely remove the screw in order to do this.
- You can then lift out the batteries (Fig. 38).

RISK OF ENTRAPMENT! Ensure that you are aware of the increased risk of entrapment when stowing your scooter.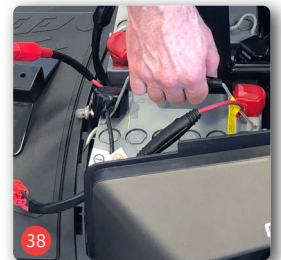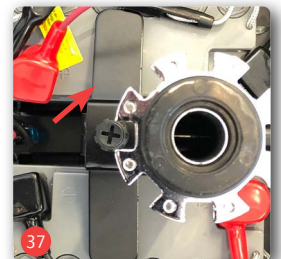

6.10. Accessories

6.10.1. Rear-view mirror right (optional)

It is also possible to fit a rear-view mirror on the right-hand side of the vehicle.

6.10.2. Cane holder for 2 walking sticks (optional)

The walking stick holder is fitted either to the seat frame or to the multifunctional holder (Fig. 18).
When mounted on the seat frame, the cane holder is secured with a retaining strap.
Never climb onto the cane holder and never use it as a tilting aid.

6.10.3. Licence plate holder (optional)

For attaching an insurance licence plate to the rear of the carrying handle of your scooter. Only for Centuro S2.
This position is recommended so that reflectors and lights are not covered.

6.10.4. Rain cover (optional)

The rain cover protects your scooter from the rain. It should be fitted when you park your scooter outdoors.
To ensure that your scooter is completely covered by the rain cover, the backrest should be folded down and the handlebars adjusted backwards.

7. Technical data

● Standard			
○ Optional		S1	S2
Abmessungen und Gewichte			
Max. load	160 kg	●	●
Seat width:	457 mm	●	●
Seat depth:	457 mm	●	●
Seat angle:	0°	●	●
Seat height from floor to front edge of seat:	628 / 653 / 678 mm	●	●
Distance of arm rest to seat surface:	230 / 208 mm	●	●
Back rest height / Back rest height with head rest:	482 mm / 665 mm	●	●
Back rest angle:	0 - 30°	●	●
Overall length:	1190 mm	●	●
Overall width:	635 mm	●	●
Pack size Length / Height:	1190 / 920 mm	●	●
Unloaden weight incl. basket and batteries:	94,5 kg	●	
Unloaden weight incl. basket and batteries:	109,5 kg		●
Weight of heaviest part:	17 kg Sitz	●	●
Elektrical system:			
Fuses:	50 A (rücks)	●	●
Overload switch resetable:			
Batteries:	2 x 12 V, 35 Ah (maintenance free, leak proof)	●	

● Standard			
○ Optional		S1	S2
Batteries:	2 x 12 V, 55 Ah (maintenance free, leak proof)		●
Charger:	24 VDC / 5 A	●	●
Motor:	DC 24 V / 550 W (6km/h)	●	●
Motor:	DC 24 V / 600 W (10km/h)		●
Controller:	PG 70A (S-DRIVE)	●	●
Max power of USB-Ports:	ca. 1 A/5 V		●
Lights and indicators:	as required by German traffic regulations (StVZO)	●	●
Tyres			
PU:		●	
Air:			●
Driving wheels:	254 /10" (2,5 bar)		●
Steering wheels:	254 /10" (2,5 bar)		●
Driving characteristics			
Speed forward:	6 km/h	●	●
Speed backwards:	10 km/h		●
Climbing capacity:	12°	●	●
Max. obstacle height:	50 mm	●	●
Range:	ca. 30 km	●	
Range:	ca. 35 km		●
Turning distance:	2050 mm	●	●
Operating forces			
Throttle lever:	< 13,5 N	●	●

● Standard			
○ Optional		S1	S2
Buttons on the controller:	< 5 N	●	●
Disconnecting the drives:	< 60 N	●	●
Operating conditions			
Temperature:	-10 - +50° C	●	●
Air humidity:	20 - 80 %	●	●
Storage conditions			
Temperatur:	-20 - +65° C	●	●
Air humidity:	45 - 60 %	●	●
Equipment			
Frame:	Pulverbeschichteter Stahlrahmen	●	●
Panel:	Kunststoff	●	●
Seat:	Kunstleder	●	●
2 independent braking systems as required by German traffic regulations (StVZO)			
Accessories: Exterior mirror right, stick holder 2 walking sticks, licence plate holder, rain cover.			

8. Care instructions for the user

8.1. Cleaning and disinfection

Cleaning

- Make sure that the scooter is switched off before starting to clean it.
- The frame components and panels of the scooter can be wiped with a damp cloth. You can use a standard mild cleaning agent if they are very dirty.
- The wheels can be cleaned with a damp brush with plastic bristles (do not use a wire brush)
- You can use a damp cloth to remove dust and minor dirt from the seat, back rest and armrests. Do not use aggressive cleaning agents because they may attack the surface of the material. The upholstery can be washed with
 - warm water at 40 °C.
- Make sure that the electrical and electronic components do not come into contact with water unnecessarily.

Disinfection

- The scooter must be thoroughly cleaned in accordance with the instructions prior to disinfection.
- Use a standard disinfectant for wipe disinfection.
- Spray the upholstery and handles.
- Allow the disinfectant sufficient time to work before wiping off, as specified in the manufacturer's instructions.
- Make sure that the electrical and electronic components do not come into contact with liquid unnecessarily.

Warning!
Do not use a high-pressure or steam cleaner.

8.2. Batteries

Your scooter is equipped with maintenance-free gel batteries whose fluid level doesn't need to be checked.

If your scooter is not used for an extended period, the charger should be connected about every 4 weeks to prevent damage to the batteries by deep discharge.

Before embarking on any journey, just check the charge status of your battery on the battery status display located on your controller. If the battery status display is in the low range, do not travel longer distances and promptly connect the battery charger to the scooter. We recommend connecting the charger to the scooter after every journey.

8.2.1. Charging

Only use approved chargers to charge the batteries (type in accordance with the technical data or as delivered by the Bischoff & Bischoff dealer Fig. 39).

Please also follow the operating instructions for the charger!

Do not charge frozen batteries.

The batteries must be charged when the temperature is between 0° and 40°C.


When using the scooter for the very first time, we recommend driving until just 3 bars are visible in the display (see Chapter 6.1) and recharge the batteries at that point.

In the event that this charge level is displayed, you should recharge the batteries as soon as possible in order to avoid deep discharge.

Recharging requires approx. 12 - 16 hours.

When using the product after that, you can recharge the batteries at any battery charge level.

You can interrupt the charging procedure even if the batteries are not fully charged, if you need to continue driving, for example.

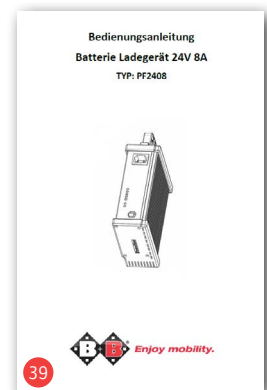But where possible, always charge to 100% so that you have the greatest range.

Charging batteries:

- First switch the scooter off.
- Before you start using the charger, check that the power supply is 230 V.
- Connect the charger with the charging socket of the scooter (Fig. 40) and then with the power socket (230 V).
- The charging socket is located on the side below the ignition switch (Fig. 41).
- Once the charging procedure is complete, unplug the mains plug and disconnect the scooter from the charger.
- When not in use, the battery should be charged once a month in order to avoid deep discharge.
- Deep discharge shortens the service life of the battery significantly, or the battery may also be damaged by the deep discharge.
- Batteries should be stored upright, in dry, frost-free conditions.
- As soon as the range with fully-charged batteries starts to decrease significantly, you should replace the batteries.

8.2.2. Replacing the batteries

Be very careful when working on the batteries. Do not use metal tools, do not touch the battery terminals and ensure that you follow the battery manufacturer's instructions.



Caution! This work should only be done by a trained technician. There is danger of fatal injury!

Only use batteries that are approved for this scooter (see Technical data).

- Make sure that the scooter is switched off before starting to replace the batteries.
- (See also Section 6.9 - Disassembling for storage).
- The batteries are located underneath the seat of the scooter and are protected by the housing cover.
- Start by removing the housing cover.
- - Disconnect the plug connections carefully and without using force.
- Undo the screw on the mounting bracket.
- Now remove the batteries (Fig. 42).
- Lift the new batteries into the space provided and connect the contacts. Make sure to observe the correct colour-coding (red and black, Fig. 43).
- Reattach the housing cover and the seat

When the batteries have reached the end of their life, i.e. cannot be sufficiently charged any more, they must not be disposed of with the household waste. When new batteries are purchased, the mobility dealer takes the old batteries and disposes of them correctly.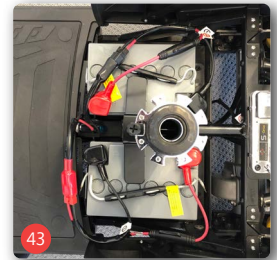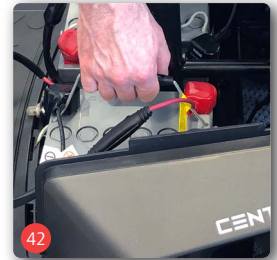

8.3. Fuses

In addition to the overload switch (see Chap. 6.6), there is also a fuse in the battery cable (Fig. 42).

9. Maintenance instructions

Typical wear parts are brakes and tyres. Contact your dealer for spare parts.

We recommend that you take your wheelchair to your dealer for inspection at least once a year.

If there are faults or defects, take the wheelchair to your dealer immediately for repair.

The information and documentation required for repair and maintenance shall be supplied to the mobility dealer on request.

If you have questions, you can contact the Bischoff & Bischoff customer service. You can find the contact details for Customer Service on the back cover of this instruction manual.

You must check your wheelchair regularly according to the maintenance schedule and find it to be in good working order.

9.1. Maintenance instructions for the Specialist trade

Maintenance must be carried out in accordance with the Bischoff & Bischoff maintenance schedule.

A maintenance plan is provided on request. A training programme for the specialist trade is also available.

10. Repair

Repairs and maintenance work may only be carried out by authorised partners (medical supply stores or specialist dealers). Please call our customer service department to find a partner in your area. The customer service contact details can be found on the back of these operating instructions.

11. Storage / Shipping

If the scooter is to be stored or shipped, all attachable and unsecured parts must be removed and packed individually in appropriate cartons. The individually packed parts can then be packaged together in a larger carton. We recommend keeping the original packaging and storing it with these purposes in mind so that it is available if required. Your scooter then has optimum protection against environmental influences during storage or transport.

When storing your scooter, please observe the information in the technical data. To prevent mould growth and damage to the upholstered parts, the storage location should be as dry as possible and not be exposed to direct sunlight.

12. Disposal

If your scooter is no longer being used and is to be disposed of, please contact your mobility dealer.

If you wish to dispose of the product yourself, contact local waste management companies to enquire about the waste disposal regulations that apply to your place of residence.

13. Use by others

The Centuro S1/S2 is appropriate for use by others. For use by others, the scooter must first be thoroughly cleaned and disinfected according to the accompanying care and hygiene instructions.

The product shall get checked by a qualified mobility dealer for general condition, wear and shall get repaired if necessary.

14. Service life

If used as intended, and the safety, care and maintenance instructions are observed, the expected service life is up to 5 years. Beyond this time the scooter can be used continuously, provided it is in a safe condition.

15. Warranty

1. For products supplied by Bischoff & Bischoff, the period of warranty is 24 months from purchase date. The products are delivered free from production and material defects. In case of evidence of material or manufacturing defect, defective parts will be replaced free of charge.

2. This warranty excludes claims, if a repair or replacement of a product or a part is required for the following reasons:

a) normal wear and tear, which include but is not limited to the following parts where fitted; Batteries, motor brushes, handles, armrests, upholstery, tyres, brakes, caps, etc.

b) overloading the product such as, for example, by exceeding the maximum user weight or load.

c) the product or part has not been maintained or serviced in accordance with the Instruction manual, the care and hygiene instructions or the manufacturer's recommendations as shown in the maintenance instructions.

d) Accessories have been used which are not original accessories.

e) The product or part has been damaged through negligence, accident or incorrect use.

f) changes/modifications have been made to the product or parts, which deviate from the manufacturer's specifications.

g) Repairs have been carried out, before our customer service has been informed of the circumstances.

3. To enforce the warranty, please contact your dealer with the exact details of the nature of the difficulty. The product must be repaired by a dealer authorised by Bischoff & Bischoff.

4. For parts, which have been repaired or exchanged within the scope of the warranty, the warranty will be extended to the remaining warranty period for the product in accordance with point 1).

5. For original spare parts which have been fitted at the customer's expense, these will have a 12 months guarantee, following the fitting.

6. If the repair fails after a reasonable time, the customer can, at his discretion, request a reduction, compensation or the termination of the purchase contract.


7. The guarantee is subject to the law of the country in which the product was purchased from Bischoff & Bischoff & Bischoff.

Please also observe the Bischoff & Bischoff care, warranty, hygiene and maintenance instructions. Your dealer will be happy to provide these to you.

For replacement devices and repairs, the warranty period is one year.

Non-observance of the operating instructions as well as improper use, maintenance jobs and technical changes or adaptations without the permission of the Bischoff & Bischoff company, lead to guarantee loss and the loss of product liability in general.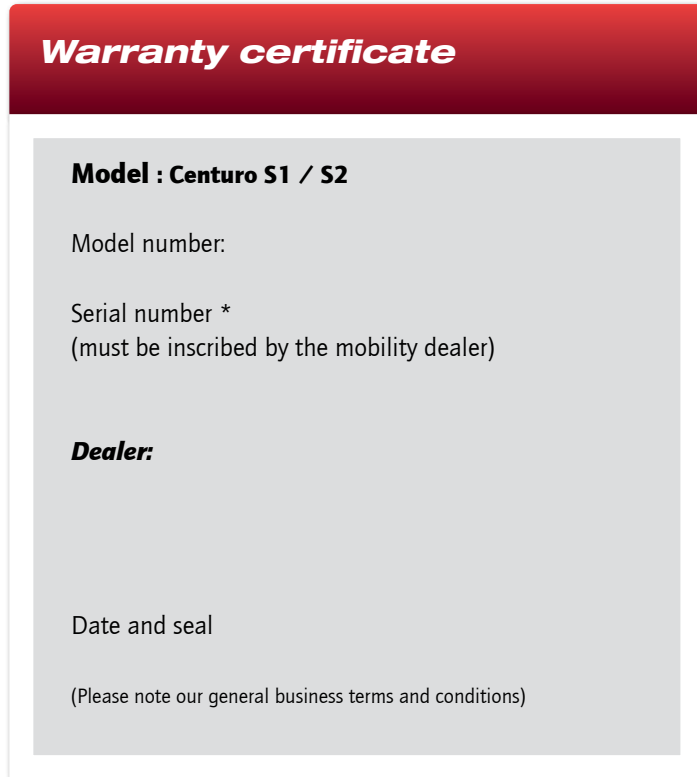

*Information labels are located at the front under the battery cover

Tip!

In spite of careful research and editing, incorrect information may have crept into our operating manuals. Rights to technical changes reserved. Illustrations may include accessories. The current version of each instruction manual is ready for download at the Bischoff & Bischoff website:

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