Bedienungsanleitung  
Instruction manual  
Manual de instrucciones  
Mode d'emploi

# **Tobago** **Tobago SL**


DE

EN

ES

FR

*Enjoy mobility.*

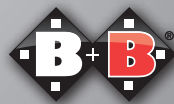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## Index

1.	Preface	21
1.1.	Signs and symbols	21
2.	Important safety instructions	22
2.1.	Indications	22
2.2.	Contraindications	22
2.3.	Service life	22
2.4.	Purpose	23
2.5.	Declaration of Conformity	23
2.6.	Responsibility	23
3.	Description of product and contents	24
3.1.	Inspection on delivery	24
3.2.	Contents	25
3.3.	Type plate and serial number	26
3.4.	Summary	27
4.	Assembly and adjustments	28
4.1.	Installing the feet and adjusting the seat height	28
4.2.	Backrest	28
4.3.	Seat	29
4.4.	Folding/unfolding	29
5.	Use	30
6.	Transport	31
7.	Storage/shipping	31
8.	Repairs	31
9.	Technical data	32
10.	Care instructions for the user	33
10.1.	Cleaning	33
10.2.	Disinfection	33
11.	Maintenance	34
12.	Disposal	34
13.	Transferral of the shower chair to another user	35
14.	Warranty	35

## 1. Preface

Dear users,

Having purchased this Bischoff & Bischoff shower aid, you now have a top German product that will simplify personal hygiene at home.

The shower chair is offered in the Tobago and Tobago SL models. The Tobago SL is easier for larger users to handle due to its higher seat position.

This instruction manual will familiarise you with the operation and use of your B+B shower chair. Please read the instruction manual carefully before using the product for the first time.

We hope you enjoy your new product!

### 1.1. Signs and symbols

**Important!** Indicates particularly useful information in the specific context.

**Caution!** Indicates information that is specifically relevant to safety. Follow the operating manual!

## 2. Important safety instructions

- Max. user weight – Tobago: 120 kg, Tobago SL: 150 kg.
- Only use the shower chair indoors and with the help of an assistant.
- Adjust all four feet to the same height.
- Ensure that the locking button is seated correctly.
- Be aware of the risk of entrapment when setting up and folding the seat.
- The seat is appropriate exclusively for sitting.
- Avoid leaning out too far to the side. Risk of tipping.
- The shower chair must be secured against unauthorised use (especially by children).

### 2.1. Indications

Unsteadiness when standing or inability to stand with reduced use or lack of function of lower extremities, (e.g. loss of strength from malignant diseases; chronic, advanced, heart, circulatory and lung diseases) combined with weak trunk stability.

### 2.2. Contraindications

The shower chair is inappropriate for persons:

- with reduced or restricted sitting ability
- with serious cognitive restrictions

### 2.3. Service life

If used as intended, and the safety, care and maintenance instructions are observed, the expected service life of the shower chair is up to 5 years. The shower chair can continue to be used beyond this period, provided the product is in a safe condition.

### 2.4. Purpose

The shower chair is designed exclusively for sitting in the shower for users for whom it is still possible to use the shower independently while sitting down.

The product is for use indoors according to the indications above. The maximum user weight of 120 kg for the Tobago and 150 kg for the Tobago SL must not be exceeded.

Please note:

A warranty claim can only be accepted if the product is used under the specified conditions and for the specified purposes.

### 2.5. Declaration of Conformity

As the manufacturer of the product B+B declares Tobago and Tobago SL meet the requirements of Directive 93/42/EEC in full.

### 2.6. Responsibility

We are able to provide warranty cover only if: the product is used in accordance with the specified conditions and for the intended purposes, if modifications, extensions, repairs, and maintenance work are carried out only by persons authorised to do so by us and if the shower chair is used in accordance with all instructions for use.

### 3. Description of product and contents

#### 3.1. Inspection on delivery

All B+B products are inspected before they leave our factory and the CE mark is applied.

The B+B shower chairs are delivered in a special box. The product is delivered and instruction in its use is given by a qualified dealer in medical products.

If the product is delivered by rail or road transport it must be inspected for shipping damage in the presence of the delivery person.

If any damage is found, please note the following:

- record the facts
- prepare a declaration of assignment; all claims resulting from the damage are assigned to the delivery company
- return the record of facts, consignment note and declaration of assignment to Bischoff & Bischoff GmbH.

#### ***In the event of shipping damage...***

- Leave the goods and packaging undisturbed and do not use the goods.
- Report the damage to the transport company in writing as instructed below.
- Contact our customer service +49 (0)700/6000 7070.
- A representative of the transport company must be informed of any damage without delay in order to make a claim for compensation.

Externally obvious damage or losses must be confirmed in writing on consignment note by the delivery person before accepting the package. Concealed damage that is not obvious until the package is opened must be reported immediately in writing to the transport company.

All deadlines must be met:

- courier service and post within 24 hours
- rail delivery within 7 days
- road transport within 4 days after receipt of the package.

Bischoff & Bischoff GmbH will require the original consignment note with confirmation of damage.

Please pay particular attention to the highlighted sections.

Compensation from the insurer is not possible without these documents.

#### 3.2. Contents

The scope of supply in the special box includes:

- 1 shower chair pre-assembled
- 1 operating manual

### 3.3. Type plate and serial number


The type plate (see fig. 01) and serial number are located under the seat.


- A** Manufacturer logo
- B** Manufacturer information
- C** Important - Follow instructions for use
- D** CE mark
- E** Model designation
- F** Model number
- G** Maximum load capacity
- H** Date of manufacture
- I** EAN 13 item number
- J** Serial number

### 3.4. Summary

To make it as easy as possible to handle your B+B shower chair, we have documented the written information with detailed illustrations in addition. The following illustrations should help clarify the names and interactions of the individual components (see figs. 1a and 1b).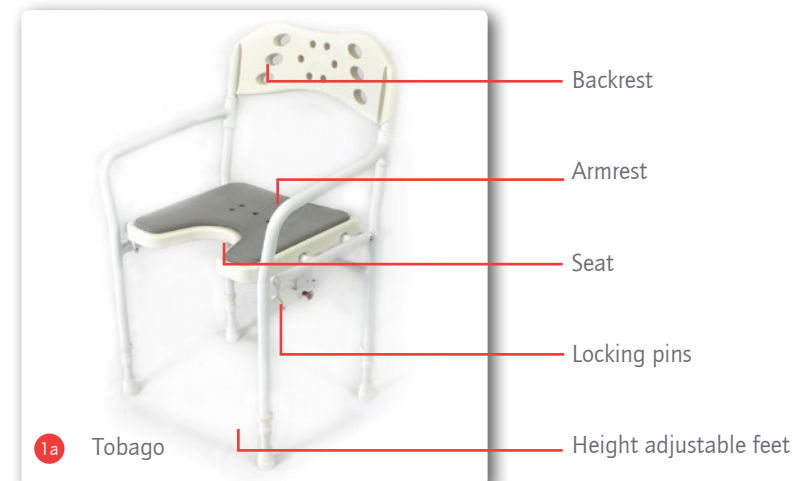

## 4. Assembly and adjustments

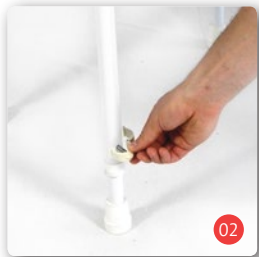
Your new B+B shower chair is pre-assembled in a B+B original box. Attached parts are supplied separately to prevent damage during transport.

You can carry out the adjustments of your shower chair yourself with a little manual dexterity. If you have any queries, please contact your medical supply dealer.

### 4.1. Installing the feet and adjusting the seat height

- Insert the foot parts into the respective frame tubes and secure them with the locking pin (fig. 02).
- Ensure that the safety bracket of the pin is closed (fig. 03).
- The rear feet of the Tobago SL are connected with a strut and are spread wider (fig. 04). The installation is the same as Tobago.
- You can adjust the seat height of the Tobago using the corresponding holes.
- Ensure that all feet are adjusted to the same height so the chair is stable.

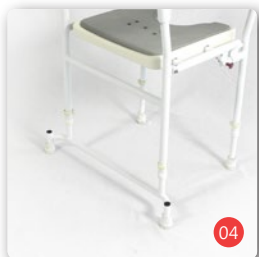### 4.2. Backrest

The backrest is placed on top of the corresponding side frame element and secured with the mounting buttons in the holes provided for them in the frame and backrest (fig. 05).

### 4.3. Seat

For initial installation the seat of the shower chair must be unfolded according to the instructions in section 4.4. The seat can then be pushed back onto the frame (fig. 06). Ensure that the recesses in the seat are located directly above the frame.

Please make sure to keep your fingers on the seat cover while unfolding the seat. Otherwise there is a risk of entrapment!

### 4.4. Folding/unfolding


#### Unfolding

- Tilt the two side frames forwards. Fold the seat frame onto the support surface of the side frame from top to bottom.
- Secure the seat and side frames by inserting the two lock pins into the holes provided (fig. 07).
- For correct seating, both grip bars and the recesses of the seat must be oriented towards the front!

#### Folding

- For folding the pressure pins under the seat the pressure pins must first be loosened. To do so, press the pressure pin with your thumb and pull the pin out at the same time (fig. 08).
- Repeat the process on the other side and use your free hand to prevent the seat from folding down.
- The seat can then be folded down, and the seat frame can be folded back (fig. 09).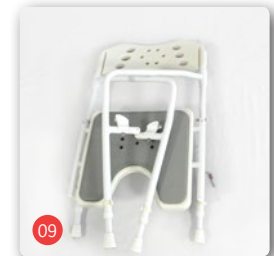

## 5. Use

Place the shower chair in the shower. Ensure that all four rubber feet are positioned securely on the floor or in the shower tray. Persons who cannot use the shower safely, should only enter it with the help of a carer.

Support yourself on the armrests when sitting down and standing up.

You can now attend to your personal hygiene. Excess water quickly drains off through the holes in the seat surface.

Remove soap residue after use and dry the shower chair off.

### ***Lifting points***

You can lift the shower chair by the metal frame in order to carry it over obstacles or stairs. Avoid lifting the chair at the seat and backrest.

## 6. Transport

To make the shower chair as convenient as possible for transport, the seat can be folded down, and the shower chair can fold up.

## 7. Storage/shipping

If the shower chair needs to be stored or shipped, it must be folded up and packed in the box. We recommend keeping the original packaging so it is available for this purpose if required. It provides optimum protection for your product from environmental influences during storage or transport.

## 8. Repairs

Repairs and maintenance must be done by authorised dealers only (medical suppliers or specialist dealers). To find a dealer in your area please contact our customer service department at +49 (0)700/6000 7070.

## 9. Technical data

### Dimensions

Width (mm):	570* 650**
Length (mm):	520
Height (mm):	880 to 955* 950 to 1030**
Seat width (mm):	450
Seat depth (mm):	400
Seat height (mm):	460 to 540 in 5 steps
Backrest height (mm):	480

### Weights

Max. weight of the user (kg):	120* 150**
Empty weight (kg):	4,8

### Materials

Frame:	Lacquered Aluminium
Seat and back:	Plastic (PE-HD)
Flammability:	flame-retardant

### Ambient conditions

Temperature:	0 to 50 °C
Air humidity:	20 to 100 %
Heat resistance:	up to 125°C

### Storage conditions

Temperature:	-20 to + 60 °C
Air humidity:	45 to 60 %

Standard equipment for: \*Tobago / \*\*Tobago SL

## 10. Care instructions for the user

### 10.1. Cleaning

Clean the material of the seats and seat backs with warm water and a sponge and/or a soft brush. A standard cleaning agent can also be used for stubborn soiling.

Note: never use aggressive cleaning agents such as solvents, scouring agents or hard brushes.

Plastic parts are best cleaned with a standard plastic cleaner. Please pay particular attention to the product information of the cleaning agent.

### 10.2. Disinfection

The shower chair can be disinfected using a general-purpose household disinfectant. Use approved spray-disinfectants or alcohol-based wipe-down disinfectants.

A list of the disinfectants approved by the Robert Koch Institute can be found at [www.rki.de](http://www.rki.de)

## 11. Maintenance

Your shower chair is made of high-quality materials for a long life of comfortable use. Every product is thoroughly checked before it leaves our premises. Nevertheless it is possible a defect may arise. If a defect arises with your shower chair, do not continue to use it. With regard to repairs, contact the medical supply dealer, who supplied the product to you. He will be pleased to assist you. We further recommend that the shower chair is serviced at least once a year. Your medical supply dealer receives a servicing schedule from us, together with the documentation required for inspections. Our Customer Service department will be pleased to answer any questions you may have with regard to process, and to offer advice and information on the free-phone number 0700/6000 7070.

You must have your shower chair regularly checked in accordance with the service schedule, and verified as being in good condition.

## 12. Disposal

If your shower chair is no longer being used and is to be disposed of, please contact your medical supply dealer.

If you wish to dispose of the product yourself, contact local recycling companies to enquire about the regulations that apply to your place of residence.

## 13. Transferral of the shower chair to another user

The shower chair is suitable for re-use. Before it is transferred to another user it should be serviced by the medical supply dealer and hygienically cleaned. When transferring the shower chair to another user, please remember to give the new user this operating manual and the address of your medical supply dealer.

## 14. Warranty

The warranty covers all defects of the shower chair that can be traced back to a material or manufacturing defect. If you make a claim under the warranty, please submit the completed warranty certificate with the claim.

We provide the statutory warranty of 2 years for this product.

Please also observe the B+B care, warranty, hygiene and maintenance instructions. Your medical supply dealer will be happy to provide you with these.

Any disregard of the operating manual, incorrectly performed maintenance work and/or technical modifications, and any additions (attachments) done without the approval of Bischoff & Bischoff GmbH shall result in the cancellation of the both the warranty and any general product liability.

**Warranty certificate**

**Product: Tobago and Tobago SL**

Serial number:\*

\* (to be entered by dealer)

**Dealer:**

Date and stamp  
(Please note our general business terms and conditions)

\*See the side frame for information stickers

**Tip!**

In spite of careful research and editing, incorrect information may have crept into our operating manuals. The current version of each of the operating manuals can be found in the download area of our website:

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### **B+B France S.A.R.L.**