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### Caneo 200 Schwerlastrollstuhl

**DIETZ**<sup>®</sup>  
**REHA-PRODUKTE**

Descostrasse 10  
D-76307 Karlsbad-Ittersbach  
Tel. +49 (0) 7248/ 9186 - 0  
Fax. +49 (0) 7248/ 9186 - 86

## Inhalt

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### Caneo 200 Heavy duty wheelchair

## Contents

<b>General information</b> .....	<b>23</b>
Preliminary remarks.....	23
Declaration of conformity .....	23
Intended purpose.....	23
Indication .....	23
Contraindication.....	23
Signs and symbols.....	24
Liability .....	24
<b>Safety instructions</b> .....	<b>24</b>
<b>Product and delivery overview</b> .....	<b>25</b>
Checking the delivery .....	25
Identification and type plates .....	25
The Caneo 200's equipment.....	26
<b>Preparing the wheelchair for use</b> .....	<b>27</b>
Folding apart (Figs. 1 - 2).....	27
Backrest surface (Fig. 3).....	27
Leg supports (Fig. 4).....	27
Foot plates (Fig. 5).....	27
<b>Basic settings</b> .....	<b>27</b>
Removing and connecting the rear wheels (Fig. 6).....	27
Seat depth (Figs. 7 - 9).....	27
Seat height adjustment (Figs. 10 - 11).....	28
Lower leg length (Fig. 13) .....	28
<b>Using your wheelchair</b> .....	<b>28</b>
Entering and exiting .....	28
<i>Entering and exiting from the side</i> .....	28
<i>Entering and exiting from the front</i> .....	28
Swivelling the side panels back (Fig. 15) .....	29
Folding the foot plates up (Fig. 16) .....	29
Swivelling away and detaching the leg supports (Fig. 17).....	29
Pushing and braking .....	29
<i>Pushing and braking by an accompanying person</i> .....	29
<i>Driving and braking independently</i> .....	29
<i>Handbrake (Figs. 18 - 19)</i> .....	30

<i>Anti-tip wheels (optional)</i> .....	30
<i>Tilting aids (Fig. 20)</i> .....	15
Transporting and folding together (Fig. 21) .....	15
Safety belt (optional) (Fig. 22).....	31
<b>Adjustment</b> .....	<b>31</b>
Locking brake (Figs. 23 - 24) .....	31
Drum brake (optional) .....	31
<b>Care instructions for the user</b> .....	<b>31</b>
Cleaning .....	31
Forwarding / reusing .....	32
Checking the wheelchair by the user .....	32
<b>Maintenance instructions for the specialist retailer</b> .....	<b>33</b>
<b>Disposal</b> .....	<b>33</b>
<b>Technical data</b> .....	<b>34</b>
<b>Maintenance log</b> .....	<b>35</b>
<b>Warranty</b> .....	<b>36</b>
<b>Warranty slip</b> .....	<b>36</b>

## **General information**

### **Preliminary remarks**

Thank you for purchasing a wheelchair from our company.

You will not be disappointed by the design, the functionality or the quality of these modern wheelchairs.

Please read these operating instructions carefully before first using the wheelchair. They contain important safety instructions and valuable tips on correctly using and caring for the wheelchair. If you have any questions or require additional information, please contact your specialist retailer which supplied the wheelchair to you.

These operating instructions contain all of the instructions necessary for adjusting and operating the Caneo 200 wheelchair. The Caneo 200 is characterised by its high stability and multiple setting options.

### **Declaration of conformity**

With sole responsibility, Dietz GmbH declares that the Caneo B wheelchair is in conformity with EU Directive 93/42 pertaining to medical products.

### **Intended purpose**

The Caneo 200 wheelchair has been designed for indoor and outdoor use and is intended exclusively for self- and third-party transportation of persons.

The Caneo 200 is designed for a maximum load of 200 kg.

### **Indication**

Inability to walk or severely pronounced walking disability due to

- Paralysis
- Loss of limbs
- Limb defect / deformation
- Joint contracture / joint damage (not on both arms)
- Other diseases

### **Contraindication**

Use of the wheelchair is unsuitable in the case of

- False sensations
- Severe dysequilibrium
- Loss of limbs on both arms
- Joint contracture / joint damage on both arms
- Inability to sit
- Impaired or inadequate vision

## Signs and symbols

	Attention! Designates particularly safety-relevant information. Observe the operating instructions!
	Important! Designates particularly useful information in the relevant factual context.

## Liability

Dietz GmbH only bears liability if the products are used under the specified conditions and for the specified purposes. We recommend that the products be handled appropriately and cared for according to the instructions. Dietz GmbH accepts no liability for damage caused by components and replacement parts which have not been approved by Dietz GmbH. Repairs must only be carried out by authorised specialist retailers or by the manufacturer itself.

## Safety instructions

To avoid falls and hazardous situations, you should first practice using your new wheelchair on level, clearly laid out terrain. An accompanying person is recommended in this case.

- Never leave children or adolescents in the wheelchair without supervision.
- Before using the wheelchair, please check whether all components are properly secured.
- Before using the wheelchair for the first time, practice driving on level, clearly laid out terrain. Intensively familiarise yourself with its braking and acceleration behaviour when driving straight ahead and on cornering.
- Please note that upholstered parts which are exposed to direct sunlight become heated and may cause skin injuries on contact. Therefore, cover these parts or protect the wheelchair from exposure to the sun.
- The wheelchair must not be misused to transport several persons or loads. Observe the intended purpose.
- Never use the wheelchair under the influence of alcohol or other substances which influence attentiveness or physical and mental receptiveness.
- Please note that the risk of tipping over may be increased due to shifts in balance as a result of body movement or loading the wheelchair.
- It is vital to use ramps to overcome obstacles.
- If they cannot be bypassed, ruts, rails or similar obstacles should be crossed at right-angles (90°).
- A risk of tipping over backwards exists as of a slope of 10°.
- The anti-tipping supports must be secured in the correct position to increase protection against tipping over backwards.
- Avoid unbraked pushing against an obstacle (step, kerb edge) as the occupant may otherwise fall out forwards.
- Never stand on the foot plates when entering or exiting.

- Make sure that the tread depth of your tyres is sufficient (>1 mm).
- Observe the road traffic regulations when driving in road traffic.
- The passive illumination on the wheelchair must always be visible to other road users.
- Wherever possible, wear light, eye-catching clothing. You will then be seen more easily by other road users.
- Do not use high-pressure cleaners or strong, corrosive chemicals for cleaning purposes.

## Product and delivery overview

### Checking the delivery

The wheelchair is completely packaged in a cardboard box. Please keep the packaging if possible; it can subsequently be used to store the wheelchair if necessary.

On receipt of the wheelchair, please immediately check to make sure that the contents are complete and undamaged. The contents consist of:

- 1 Cardbord box
- 1 pre-assembled wheelchair
- 1 padded back upholstery
- 1 pair of leg supports, footboard and heel strap
- 1 set of operating instructions
- 1 set of tools

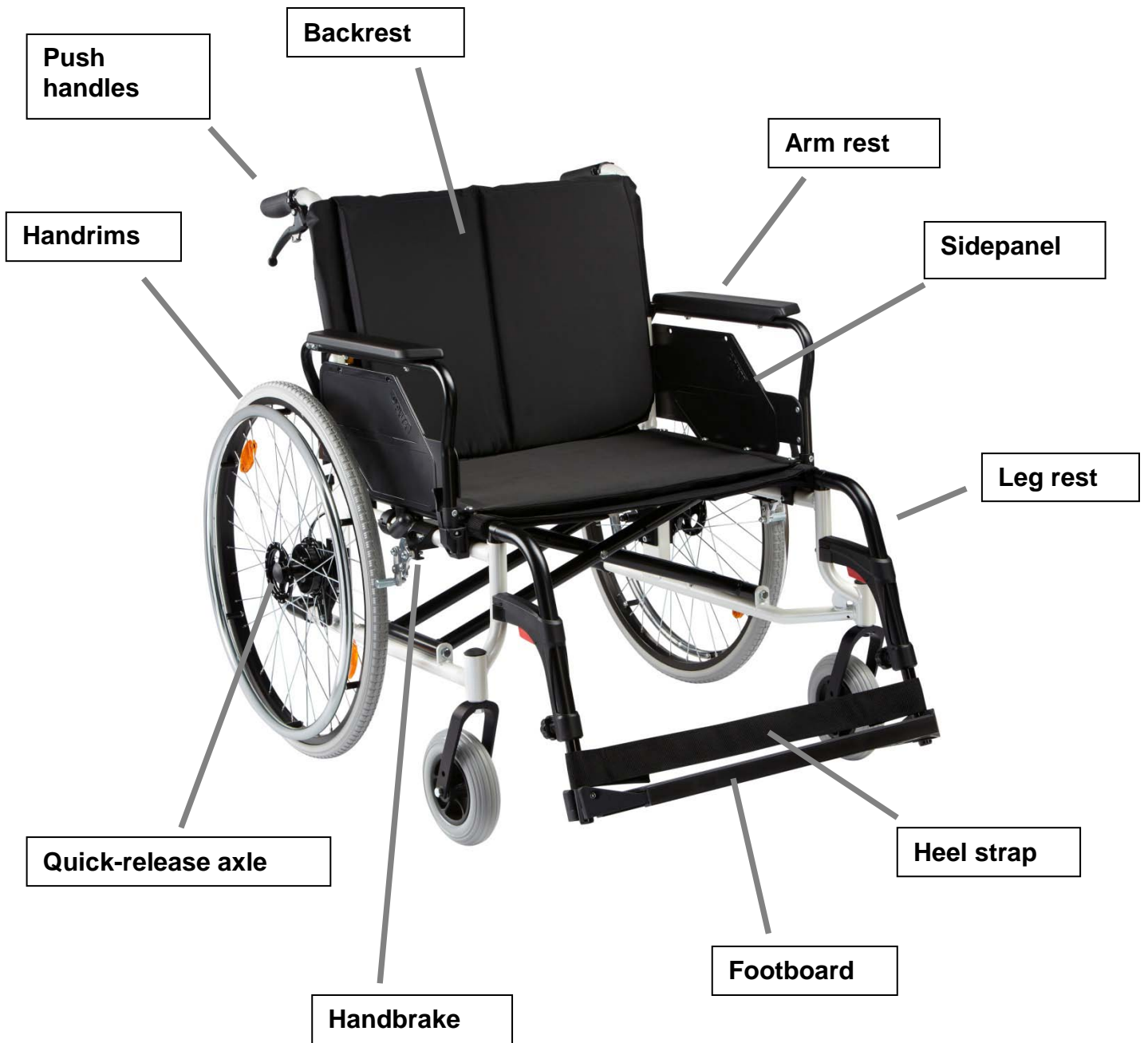
### Identification and type plates

The type and manufacturer's plates are mounted on the frame beneath the seat.

The plates must remain legible. Immediately replace illegible or missing plates. The type plate is very important for identifying the product. It must not be removed.

## The Caneo 200's equipment

## Preparing the wheelchair for use

Your wheelchair is usually delivered fully-assembled and folded. A few simple steps are sufficient to prepare it for use.

**Risk of trapping!**

When folding and unfolding/erecting your wheelchair, pay attention to your fingers and only grip the parts which are illustrated.

### Folding apart (Figs. 1 - 2)

Position yourself next to the wheelchair and tilt it slightly towards you. Then press the crossbar apart with the flat of your hand. It might be necessary to loosen the backrest Velcro straps.

Make sure that the crossbar engages in the rests.

### Backrest upholstery (Fig. 3)

Lay the back upholstery over the adjustable Velcro backrest and pull the bib beneath the seat and secure it with the Velcro strap; this closes the gap between the backrest and the seat surface.

### Leg rests (Fig. 4)

Hook the leg supports in, put on the and swivel them forwards.

### Foot plates (Fig. 5)

Finally, put on the footboard, fold the foot plates down and attach the heel strap.

## Basic settings

### Removing and connecting the rear wheels (Fig. 6)

During many adjustment operations, it is necessary to remove the rear wheels.

Press the button on the quick-release axle and reach into the spokes with four fingers. Then pull the rear wheel (the quick-release axle) out of the axle mounting.

To connect the rear wheels, press the quick-release axle button with your thumb again. Now insert the rear wheel (the quick-release axle) into the axle mounting; the quick-release axle engages automatically.

### Seat depth (Figs. 7 - 9)

The wheelchair is always delivered with the maximum possible seat depth. To reduce the seat depth, first remove the leg rests. Next, open the Velcro straps on the right and left of the seat frame. Secure the Velcro straps to the extension so that they are no longer wrapped around the frame. Fold the extension beneath the seat and fasten the Velcro straps to secure it there.

Carry the procedure out in the opposite sequence to increase the seat depth.

## Seat height adjustment (Figs. 10 - 11)

The height of the wheelchair's seat is adjusted in 2 stages of 30 mm (47, 50 cm). The front fork has 2 different holes to set the front wheel height. The height of the rear wheel is adjusted by moving the wheel adapter plate up or down. Loosen the securing bolts to do this. Adjust the rear wheel adapter to the desired height and tighten the bolts again.

A change in the height of the seat necessitates readjustment of the brakes.

## Lower leg length (Fig. 12)

The setting is correct when your upper legs are horizontal when your feet are positioned on the foot plates.

To adjust, release the threaded connection one quarter of a turn and pull the locking mechanism backwards. Move the foot plate to a comfortable position and release the locking mechanism again. The mechanism automatically locks in the next position. Then tighten the threaded connection securely again.

## Using your wheelchair

### Entering and exiting

#### *Entering and exiting from the side*

The wheelchair can be easily entered and exited. The foot plates, arm rests and leg rests can be swivelled up or down.

Move the wheelchair and your current or future seat as closely together as possible.

If you wish to transfer to the wheelchair from another wheelchair or another seat, secure the wheelchair to prevent it from rolling away by actuating the handbrakes.

Swivel the side panel to the rear and fold the foot plates up.

Swivel the leg rests inwards or outwards (if necessary, on both wheelchairs if you wish to transfer from one wheelchair to another).

Now slide sideways onto the seat surface.

Please make sure that you are sitting as far back in the seat as possible.

Then swivel the side panel and the leg rests back to their original position.

#### *Entering and exiting from the front*

Fold the foot plates up and swivel the leg rests away outwards. (Description, see above).

Move the front of the wheelchair and your current or future seat as closely together as possible.

If you wish to transfer to the wheelchair from another wheelchair or another seat, secure the wheels by actuating the handbrakes.

Now slide onto the seat surface by turning your body. Please make sure that you are sitting as far back in the seat as possible.

Then swivel the leg rests back to the centre, fold the foot plates down and place your feet onto them.

## Swivelling the side panels back (Fig. 13)

To swivel up, press the detent lever and pull the side panel upwards.  
The arm rest engages automatically on swivelling back down.  
This engagement is clearly audible.

## **Folding the foot plates up (Fig. 14)**

Fold the foot plates up to make entering and exiting easier.  
The foot plates must be folded up, and the footboard and heelstrap have to be removed when folding the wheelchair together.

## **Swivelling away and detaching the leg rests (Fig. 15)**

To remove the leg rests, first detach the heel strap, fold the foot plates up and remove the footboard.

Now actuate the locking mechanism (red) and swivel the leg rest outwards.

The leg rest can now be detached upwards.

Carry the procedure out in the opposite sequence during assembly.

When the leg rest is swivelled forwards, it can be clearly heard to engage.

## **Pushing and braking**

### ***Pushing and braking by an accompanying person***

The accompanying person propels the wheelchair by pushing on the intended pushing handles.

**Important!**

In models without an optional drum brake, the accompanying person must be physically and mentally able to brake the wheelchair by using his own strength. The locking brake is then available as a parking brake.

The accompanying person can use an optional drum brake as a service and locking brake. This helps when moving on slopes. Movement can then be dosed and braked by applying the brake levers more or less extensively subject to requirements. In addition, the drum brake can also be used as a parking brake.

### ***Driving and braking independently***

Carry out initial test drives carefully on flat terrain with the support of an assistant until you have become familiar with your new wheelchair. You may only operate the wheelchair in public road traffic once you have become accustomed to operation. You must take particular care when driving on inclines and declines or when overcoming obstacles.

The wheelchair is moved and braked using the handrims; when doing so, the hand is placed onto the handrim, in which case only the thumb and the curved index finger come into contact with the rim. The other fingers are closed to make a fist and do not come into contact with the handrim.

To brake the wheelchair, you should allow the handrim to slide between your thumbs and curved index fingers, and brake until stationary by gradually gripping the handrim.

**Attention!**

When propelling your wheelchair, make sure that you do not touch the tyre casing with your thumb.  
Risk of trapping between the tyre and side panel/arm rest!

To steer the wheelchair, the push rim is braked on the side to which you would like to steer or turn.

To turn on the spot, one hand pushes forwards and the other hand pushes backwards at the same time.

### ***Handbrake (Figs. 16 - 17)***

The Handbrake enables the user to park the wheelchair securely.

Actuate the handbrake (tyre brake) when stationary. To do this, push the lever forwards until it stops independently. The wheelchair is now standing securely; you can now transfer or simply park. The handbrake is released by pulling the lever backwards.

### ***Anti-tip wheels (optional)***

The optional anti-tip wheels prevent the wheelchair from tipping backwards. They are additionally used to facilitate tilting by an accompanying person.

The height of the wheels can be adjusted. To do this, pull the silver-coloured disks towards you and slide the tube upwards or downwards along the bracket.

### ***Tilting aids (Fig. 20)***

The tilting aids enable the accompanying person to tilt the wheelchair and therefore move it over obstacles more easily. First push the tilting wheels to the highest position. Place your foot onto the lever and maintain balance using the push handle.

Anti-tip supports can be optionally fitted instead of the rubber buffers; these can be used as tilting aids and tipping protection to the rear.

### **Transporting and folding together (Fig. 21)**

Your wheelchair can be comfortably folded together for transportation in a passenger car or simply to save space. It then takes up very little space and can be easily stowed.

In addition, the rear wheels can also be easily removed.

Attention!

The wheelchair is not approved for use as a seat in vehicles such as handicapped person transportation vehicles!

Leave the wheelchair and fold the foot plates upwards. Detach the leg rests and release the backrest bib beneath the seat surface. Release the backrest's Velcros straps. Now pull the seat cover up centrally; then wheelchair subsequently folds together automatically.

To erect/unfold the wheelchair, refer to the instructions under "Preparing the wheelchair for use".

### **Safety belt (optional) (Fig. 22)**

An optional safety belt can be installed. This secures and stabilises the person seated in the wheelchair. Tipping forwards out of the wheelchair is prevented.

The safety belt is secured to the backrest tube at the rear right and left. Subsequent installation should be carried out in a specialist workshop.

## Adjustment

Brake adjustment should only be carried out by trained specialists. If you have sufficient experience, however, you may also carry this work out yourself.

Attention! Incorrectly adjusted brakes may have life-threatening consequences.

## Handbrake (Figs. 23 - 24)

The distance between the brake pin and tyre casing must be 12-14 mm at the narrowest point when the brake mechanism is opened to the maximum setting.

Loosen the connecting bolt to adjust/correct the distance. Then push the brakes to the correct position. Subsequently retighten the bolt and check the function of the brake. It should be impossible to push the wheelchair when the brake is actuated.

## Drum brake (optional)

To achieve an optimal braking effect, the braking force is adjusted at the setting screw on the brake cable. The braking force is increased by unscrewing the setting screw. Loosen the locknut and unscrew the setting screw until scraping noises are heard at the rotating wheel. Then screw the setting screw in again until the scraping noises disappear. On completion of adjustment, the setting screw is secured in position by tightening the locknut.

Attention!  
Ensure even adjustment on both sides of the wheelchair.

## Care instructions for the user

### Cleaning

- The wheelchair's frame parts can be wiped down with a damp cloth. A mild cleaning agent can be used in the case of more severe soiling.
- The wheels can be cleaned using a damp brush with plastic bristles (do not use a wire brush!).
- The upholstery materials can be washed with a mild soap solution up to 40°C and can be disinfected using alcohol-based disinfectants if necessary.

Do not use high-pressure or steam cleaners!

## Forwarding / reusing

In the event that the wheelchair is forwarded / reused, please remember to hand over all technical documents required for safe handling to the new user.

The wheelchair must be cleaned and disinfected, and must be checked for damage and approved by the specialist retailer.

## Checking the wheelchair by the user

Prior to each journey, check

- that the tyres are not damaged and that the inflation pressure is adequate (with optional pneumatic tyres). Regularly check the tyre tread depth once per month. This should be at least 1 mm. Defective or worn tyres must be replaced.
- the functional capability of the brake system. It should be impossible to push the wheelchair when the brake is actuated. It should be easy to push the wheelchair when the brake is released. If you notice unusual braking behaviour, immediately notify your specialist retailer and cease using the wheelchair.
- the functional capability of the passive illumination system. Your wheelchair is equipped with reflectors. The reflectors must be visible to other road users and should not be soiled or otherwise covered.
- that the seat and backrest upholstery is undamaged and clean. If it is soiled, it can be cleaned in accordance with the cleaning instructions. If it is damaged, contact your specialist retailer.
- that all moving parts run smoothly and function with little noise. Squeaking noises indicate damage and should be checked by the specialist retailer.
- that all threaded and riveted connections are undamaged.

## Maintenance instructions for the specialist retailer

We recommend having the wheelchair inspected by a specialist retailer at least once per year. In the event of malfunctions or defects on the wheelchair, it must be immediately brought to a specialist retailer for repair.

The specialist retailer will be provided with the information and documents required for repair and maintenance on request.

If any questions arise, Dietz Customer Service can be reached by calling +49

(0) 7248/ 9186 - 0.

Please use the enclosed maintenance log to maintain an overview of the wheelchair's maintenance status.

## **Disposal**

If your wheelchair is no longer used and is to be disposed of, please contact a local disposal company to enquire about the regulations.

## Technical data

Caneo 200		
Dimensions and weights		
Seat width cm	56	60
HMV (German medical aids and appliances number) No.		
Seat height cm	44 / 47	
Seat depth cm	40 / 44	
Overall width cm	75	79
Seat angle °	3°	
Overall length cm	113 / 90 (without leg rests)	
Overall height cm	89	
Backrest height cm	40 / 43	
Back angle °	3°	
Pushing handle height cm	88 @ seat height 47	
Arm rest height cm	23	
Arm rest width cm	6	
Arm rest length cm	37	
Lower leg length cm	42,0 / 43,5 / 45,0 / 46,5 / 48,0 / 49,5 / 51	
Weight kg	27	
Max. user weight kg	200	
Rear wheels	24 x 1 3/8 heavy duty	
Front wheels	7 x 1 3/4	
Tipping stability °	Up to 10°	
Materials °	Steel frame, high-quality plastic covers	

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Please list all service and maintenance work in form below.

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Warranty services refer to all product defects which are verifiably attributable to material or manufacturing flaws. The warranty period is 24 months as of receipt of notification of readiness for shipping, but following delivery at the latest. Damage which has occurred due to natural wear, intent, negligent or improper operation or usage is excluded from the warranty obligation. This also applies on use of unsuitable care products, lubricating oils or greases. The completed warranty slip (see below) must also be submitted in the event of complaints.

✂

### Garantiekarte / Warranty slip

**Produkt  
Product:**

Modell / Model

Seriennummer / Serial number

**Fachhändler  
Specialist  
retailer:**

Datum und Stempel / Date and stamp

Senden an / Send to:

**DIETZ**<sup>®</sup>  
**REHA-PRODUKTE**

Descostrasse 10

D-76307 Karlsbad-Ittersbach